


NITRO HD BLACK

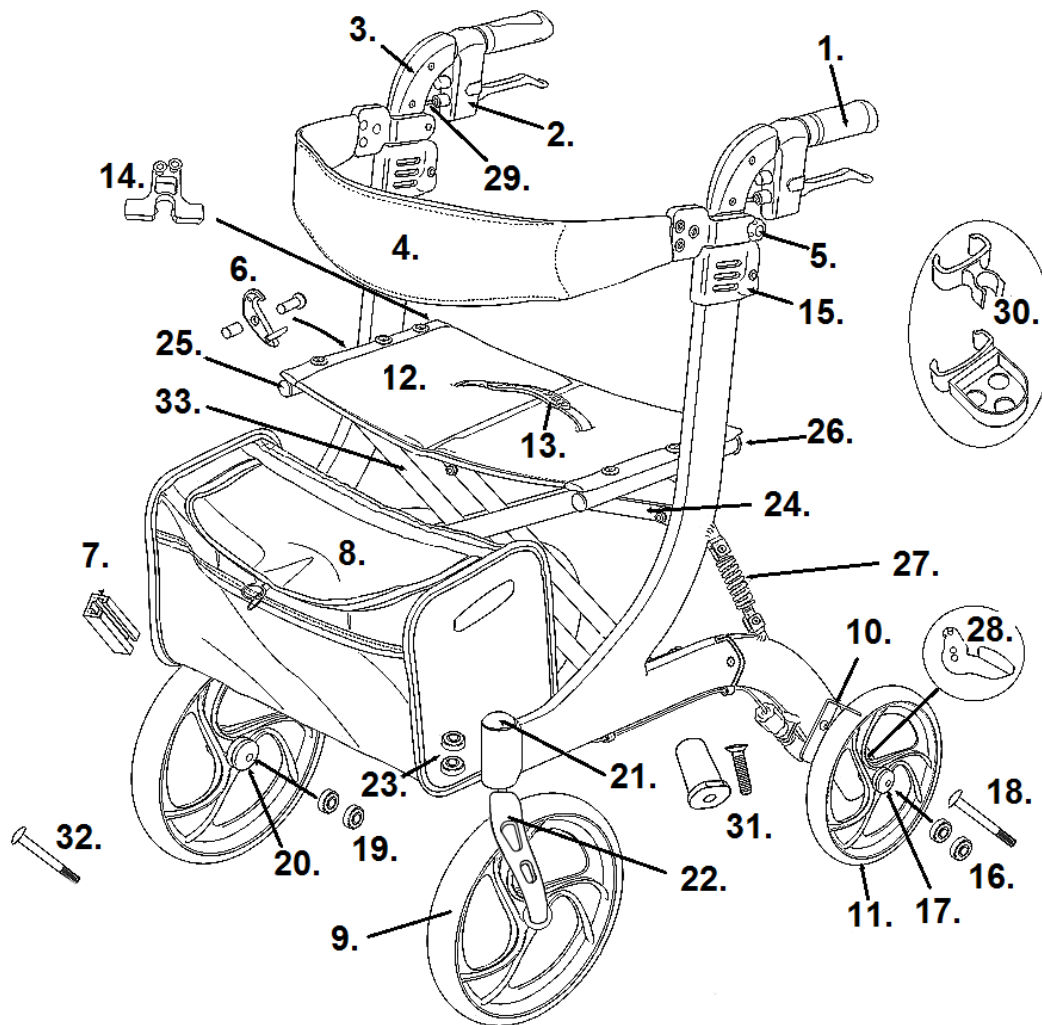

- | | |
|--|---|
| 1. Handgrip Left- 1026602
Handgrip Right- 1026603 | 15. Back Height Adjustment Lever- 1026607-R |
| 2. Hand Brake Assembly Left- 10266HD01-L
Hand Brake Assembly Right- 10266HD01-R
Cable Only (51" Long)- 10266HD02 | 16. Rear Wheel Bearing (pr)- 1026613 |
| 3. Handle Bar- 1026630N | 17. Rear Hub Cap (pr)- 1026619 |
| 4. Backrest Assy.- 10266HD04 | 18. Rear Axle- 1026631 |
| 5. Backrest Knob & Bolt (pr)- 1026605-G | 19. Front Axle Bearing (pr)- 1026618-AN |
| 6. Safety Lock- 1026638 | 20. Front Hub Cap Fork- 1026614-AN |
| 7. Bag Mount & Screw (pr)- 1026611 | 21. Dust Cap (pr)- 1026620 |
| 8. Bag- 10266HD10 | 22. Fork Assembly Left- 1026622-AN
Fork Assembly Right- 1026621-AN |
| 9. Caster & Fork Assembly Left- 1026625-AN
Caster & Fork Assembly Right- 1026624-AN
Caster Only- 1026617-AN | 23. Fork Stem Bearings (pr)- 1026623 |
| 10. Wheel Lock Assembly Plastic Left- 1026641
Wheel Lock Assembly Plastic Right- 1026642 | 24. Cross Frame Assist (pr)- 10266HD32 |
| 11. Rear Wheel Left- 1026612-L
Rear Wheel Right- 1026612-R | 25. Front End Cap (pr)- 1026628 |
| 12. Seat Upholstery & Hardware- 10266HD08 | 26. Rear End Cap (pr)- 1026628 |
| 13. Folding Strap- 1026609-FSG | 27. Shock Absorber- 10266HD31 |
| 14. Folding Locking Latch- 1026644 | 28. Step Tube Left- 1026636
Step Tube Right- 1026637 |
| | 29. Brake Cable Guide- 1026639 |
| | 30. Cane Holder- 892 |
| | 31. Cross Brace Screw & Insert- 1026643 |
| | 32. Front Axle- 1026645 |
| | 33. Cross Frame- 10266HD-33
Cross Frame Bolt- 1026646 |