


726 & R726

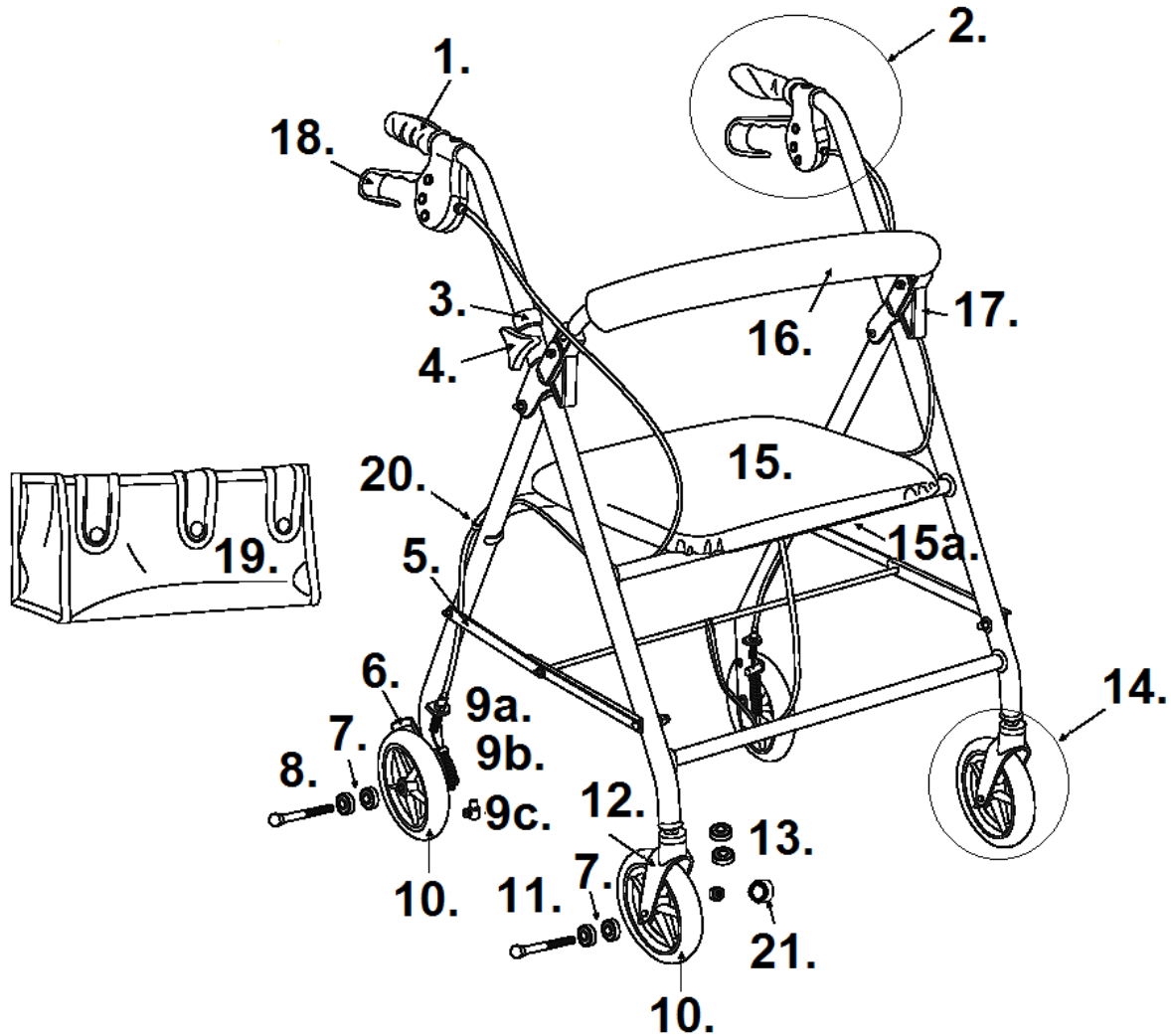

- | | |
|---|--|
| 1. Handgrip- 9501S72602 | 13. Fork Stem Bearing- STDS1060 |
| 2. Handlebar Assembly Left- [2]
Handlebar Assembly Right- [2]
Handlebar Assembly Left- 9509H72605L [3]
Handlebar Assembly Right- 9509H72605R [3]
(inc. Brake, Grip & Handlebar) | 14. Fork-Caster Assy. 1S Serial #- [2]
Fork-Caster Assy. 2M, 9H & 20H Serial #- 9502M72601
Fork-Caster Assy.1P Serial #- 9501P72601 |
| 3. Anti-Rattler- 950300169901 | 15. Seat- [2]
Seat- 9509H72604 [3] |
| 4. Height Adj. Knob- 10208KN | 15a. Seat Brackets 1S Serial #- 9505W1026104
Seat Brkt. 2M, 1P, 20H & 9H S/N- 950340059901-2M [4] |
| 5. Folding Hinge Assembly- 9501S72609 | 16. Backrest Red- [1]
Backrest Blue- [1]
Backrest Black- 9501S726BK |
| 6. Wheel Lock Right- 9501S7261R
Wheel Lock Left- 9501S7261L | 17. Backrest Pivot Bracket- 9501S72603 |
| 7. Wheel Bearing- STDS1011 | 18. Hand Brake & Cable- [2]
Hand Brake & Cable Left- 9509H72607L [3]
Hand Brake & Cable Right- 9509H72607R [3] |
| 8. Rear Axle- 9501A79506-2M | 19. Tote Bag- 9505W1026120 |
| 9a. Cable Adjuster- 9500001232 | 20. Frame Clip- 10208C |
| 9b. Spring- 9501S72616 | 21. Axle Cap- 950330149901 [4]
[1] Stock is gone use 9501S726BK
[2] These parts are no longer be available.
[3] 20H & 9H serial number
[4] When stock is gone the part is no longer available. |
| 9c. Cable Fastener- 9500004101 | |
| 10. Wheel- 9501S72600 | |
| 11. Front Axle Chrome- STDS2A3423
Front Axle Black- STDS2A3423-BLK | |
| 12. Fork Assembly 1S Serial #- [2]
Fork Assembly 2M, 9H & 20H Serial #- 9502M72610
Fork Assembly 1P Serial [2] | |