


10257

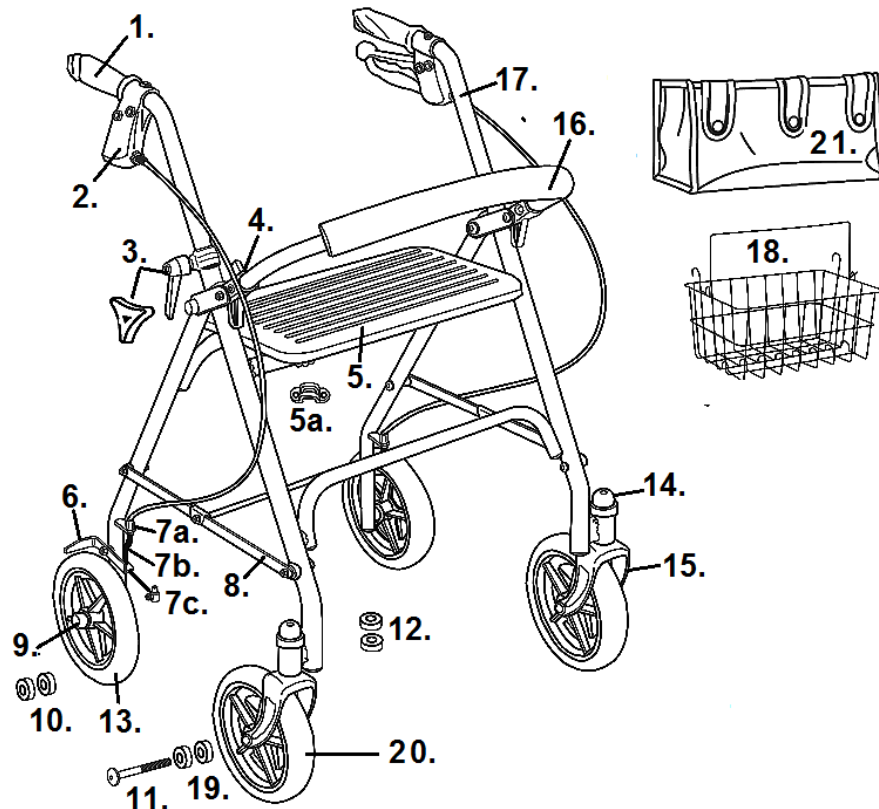

- | | |
|--|---|
| <p>1. Hand Grip Left- 95032F260N15 [4]
Hand Grip Right- 95032F260N14 [4]
2. Hand Brake & Cable- 9502F1025701
3. Height Adj. Ratchet (pr)- 95032F260N10
Height Adj. Knob- 10208K [1]
4. Backrest Bracket (pr)- 9501S72603
5. Hard Seat & Brackets- 9502F1025703 [4]
Padded Seat & Brackets- 9502F1025712
5a. Hard Seat Brackets (pr)- 9502F1025702 [4]
Padded Seat Brackets (pr)- 9502F1025713
6. Wheel Lock Left- 9502F1025710
Wheel Lock Right- 9502F1025711
7a. Cable Adjuster- 9500001232
7b. Cable Spring- [5]
7c. Cable Fastener- 9500004101
8. Folding Hinge Assembly- 9502F1025707
9. Axle Cap- STDS001000
10. Rear Wheel Bearing- STDS1011
11. Caster Axle- 9502F1025709
12. Fork Stem Bearing- STDS1060</p> | <p>13. Wheel Gray- 95032F260N09
Wheel Black- 95032F260N09-BLK
14. Dust Cap (pr)- 95032F260N06
15. Fork Blue- 9502F1025708
Fork Red- 9502F1025708R
16. Backrest- 9502F1025705
17. Handlebar Only Left- 9502F1025715 [3] [4]
Handlebar Only Right- 9502F1025714 [3] [4]
Handlebar Assy. Left- 9503Z1025715 [1] [2]
Handlebar Assy. Right- 9503Z1025714 [1] [2]
18. Basket- 9502F1025706
19. Caster Bearing- STDS1011
Caster Bearing- 1992M-19 [1]
20. Caster Gray- 95032F260N09 [4]
Caster Black- 95032F260N09-BLK [4]
Caster Black- 9503Z1025709 [1]
21. Tote Bag- 9505W1026120
2023 and later- 9503Z1025716
Not Shown Tray- [5]</p> |
|--|---|

[1] 3Z serial number.
[2] Includes parts 1 and 2. Handle height is 35".
[3] Handle height is 37".
[4] When stock is gone the part is no longer available
[5] The part is no longer available.