


# Nitro Hemi

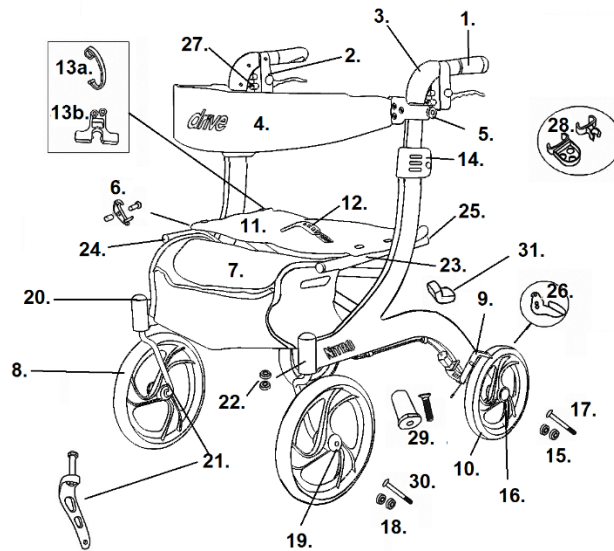

- |  |  |
|--|--|
| 1. Handgrip Left- 1026602<br>Handbrake Right- 1026603  | 12. Folding Strap Assy. 10266-H- 1026609-FSR-HEMI [6]<br>F Strap Assy. 10266BK-H- 1026609-FSG-HEMI [1] [6]<br>F Strap Assy. 10266WT-H- 1026609-FSW-HEMI [7]                          |
| 2. Hand Brake RTL10266-H Lt- 1026601-RN-HEMI [6]<br>Hand Brake RTL10266-H Rt- 1026601-RNR-HEMI [6]<br>Hand Brake RTL10266BK-H Lt- 1026601-G-HEMI [6]<br>Hand Brake RTL10266BK-H Rt- 1026601-GR-HEMI [6]<br>Hand Brake RTL10266WT-H Lt- 1026601-W-HEMI [6]<br>Hand Brake RTL10266WT-H Rt- 1026601-WR-HEMI [6]<br>Cable Only 42 3/8"- 1026632-HEMI [6] | 13a. Folding Locking Latch 10266-H- 1026626-R<br>Folding Locking Latch 10266BK-H- 1026626-G<br>Folding Locking Latch 10266WT-H- 1026626-W  |
| 3. Handlebar- 1026630N-HEMI [6]  | 13b. Folding Locking Latch- 1026644 [3]  |
| 4. Backrest Assembly RTL10266-H- 1026604-B<br>Backrest Assembly RTL10266WT-H- 1026604-B<br>Backrest Assembly RTL10266BK-H- 1026604-G   | 14. Back Height Adj. Latch RTL10266-H 1026607-R<br>Back Height. Adj. Latch RTL10266BK-H- 1026607-G<br>Back Height Adj. Latch RTL10266WT-H- 1026606-W                                 |
| 5. Backrest Knob & Bolt RTL10266-H- 1026605-R<br>Backrest Knob & Bolt RTL10266WT-H- 1026605-W<br>Backrest Knob & Bolt RTL10266BK-H- 1026605-G  | 15. Rear Axle Bearings (pr)- 1026613   |
| 6. Safety Lock- 1026638  | 16. Rear Hub Cap- 1026619  |
| 7. Tote Bag RTL10266-H- 1026610-R-HEMI [6]<br>Tote Bag RTL10266BK-H- 1026610-G-HEMI [6]<br>Tote Bag RTL10266WT-H- 1026610-W-HEMI [6]   | 17. Rear Axle- 1026631   |
| 8. Caster & Fork Assy. Style #1 Left- 1026625<br>Caster & Fork Assy. Style #1 Right- 1026624<br>Caster & Fork Assy. Style #2 Lt- 1026625-AN [5]<br>Caster & Fork Assy. Style #2 Rt- 1026624-AN [5]<br>Caster Style #1- 1026617<br>Caster Style #2- 1026617-AN  | 18. Front Axle Bearings (pr)- 1026618<br>Front Axle Bearings (pr)- 1026618-AN [4]  |
| 9. Wheel lock Assembly Metal Left- 1026615<br>Wheel lock Assembly Metal Right- 1026616<br>Wheel lock Assembly Plastic Left- 1026641 [2]<br>Wheel lock Assembly Plastic Right- 1026642 [2]  | 19. Front Hub Cap Old Style Fork- 1026614<br>Front Hub Cap New Style Fork- 1026614-AN [4]  |
| 10. Rear Wheel Left- 1026612-L<br>Rear Wheel Right- 1026612-R  | 20. Dust Cap (pr)- 1026620   |
| 11. Seat/Hdwr. (Black) RTL10266-H- 1026608-HEMI [6]<br>Seat/Hdwr. (Grey)RTL10266WT-H- 1026608-HEMI-G [6]<br>Seat/Hdwr. (Grey)RTL10266BK-H- 1026608-HEMI-G [6]  | 21. Fork Assembly Old Style Left- 1026622<br>Fork Assembly Old Style Right- 1026621<br>Fork Assembly New Style Left- 1026622-AN [4]<br>Fork Assembly New Style Right- 1026621-AN [4] |
|  | 22. Fork Stem Bearings (pr)- 1026623   |
|  | 23. Cross Frame- 1026627-HEMI [6]<br>Cross Frame Bolt- 1026646   |
|  | 24. Cross Frame Front End Cap (ea)- 1026634  |
|  | 25. Cross Frame Rear End Caps (pr)- 1026628  |
|  | 26. Step Tube Left- 1026636 [1]<br>Step Tube Right- 1026637 [1]  |
|  | 27. Brake Cable Guide- 1026639   |
|  | 28. Cane Holder- 892   |
|  | 29. Cross Brace Screw & Insert- 1026643  |
|  | 30. Front Axle- 1026645 [4]  |
|  | 31. Seat Rail Pad- STDS2A1807  |

[1] Serial # 2A140801025... & higher

[2] Serial # 2A1510302680 & higher

[3] Serial # 2A1606300234 & higher

[4] Serial # 2A1606301542 & higher

[5] Serial # 2A1606301542 to 2A1909314949

[6] When stock is gone the part is no longer available.

[7] The part is no longer available.