

maintenance

Drive recommends periodic visual inspection of this product by the user, to ensure that all parts and hardware are secure, that components are in good working order and not worn, torn, frayed, or loose, and that there are no obstructions or impediments to normal, safe operation. If any part or mechanism appears to be unsafe or compromised, do not use the product. If you experience problems, please contact the provider that sold you the product, or Drive Technical Support at (866) 621-4933.

lifetime limited warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for the lifetime of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.



© 2018 Medical Depot, Inc. All rights reserved.

Drive is a trademark of Medical Depot, Inc.

Port Washington N.Y. 11050 USA Made in China

universal crutch/walker platform attachment

item #:

10105-1, 10105-2, 10105HD-2



EU Authorized Representative
Drive Medical LTD
Ainley's Industrial Estate
Elland, West Yorkshire,
United Kingdom HX5 9JP

www.drivemedical.com

A large, vertical version of the Drive logo, with the word "drive" written in white lowercase letters on a dark blue background. The "i" and "v" have yellow accents. A small "TM" trademark symbol is at the top right of the "e".

instructions

1. Determine which side(s) of the walker the platform attachment will be used.
2. Attach armrest and handle to vertical platform post using the supplied bolts and adjust as seen in image 1. Assure that the vertical platform post is **inside** of the platform as seen in image 1.
3. Place both platform clamps on the vertical platform post with the longer clamp on first (top) from the outside towards the inside (image 2).
4. Loosely attach the bottom (smaller) clamp to the lower horizontal walker crossbar from the outside towards the inside (image 2).
5. Loosely attach the top (large) platform clamp to either the upper horizontal walker crossbar or over the top bar handle from the outside towards the inside (image 2).
6. Adjust platform post to desired height (image 3).
7. Tighten both clamps carefully adjusting both sides of clamps equally.
8. Ensure that the platform attachment is on the inside of the L bracket (image 3).
9. Adjust arm support and handle to increase patient comfort and support. Tighten all adjustment knobs to ensure patient safety.

