


## SILVER SPORT RECLINER

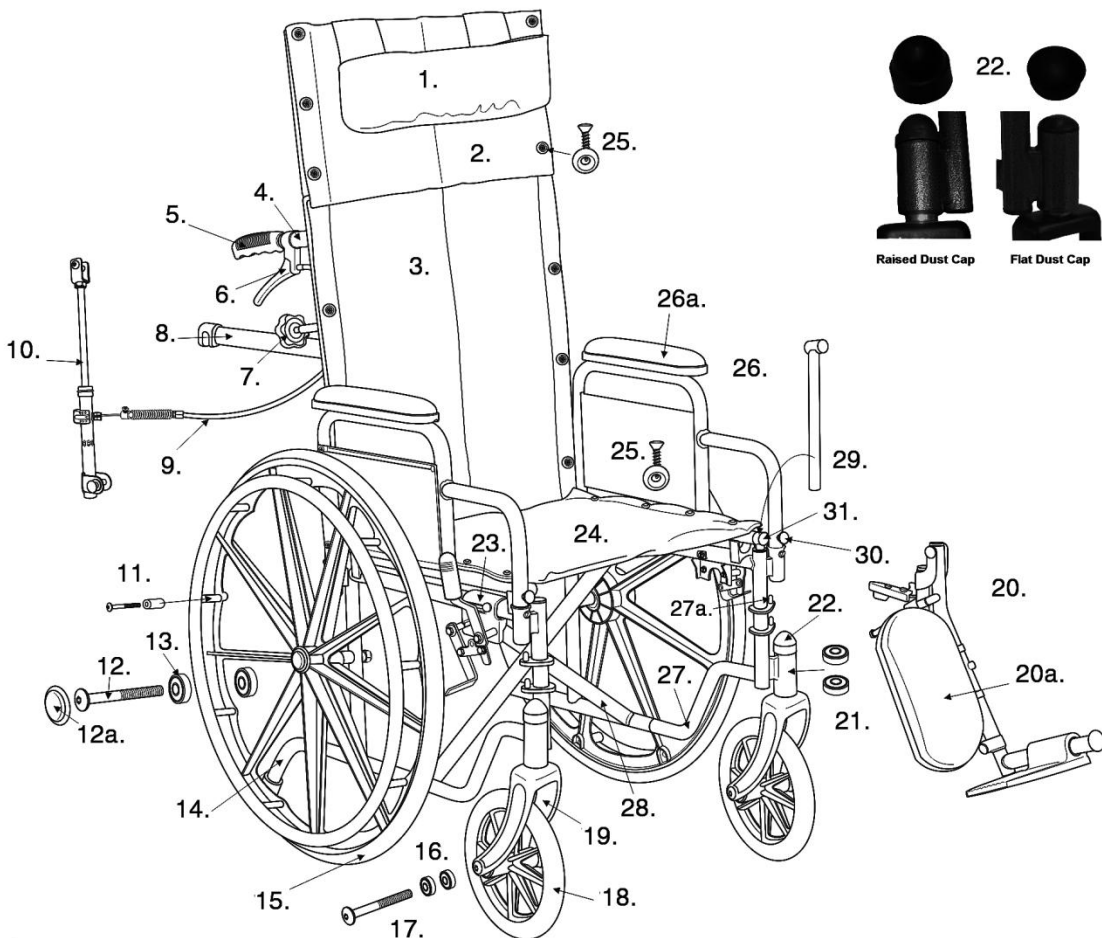

- |   |  |
|---|--|
| <p>1. Headrest Pillow- STDS4Y4726</p> <p>2. Headrest Upholstery 16"- STDS4Y4724<br/>Headrest Upholstery 18"- STDS4Y4713<br/>Headrest Upholstery 20"- STDS4Y4725</p> <p>3. Back Upholstery 16"- STDS4Y4722<br/>Back Upholstery 18"- STDS4Y4700<br/>Back Upholstery 20"- STDS4Y4718</p> <p>4. Backrest Cane S s/n- STDS4S4716<br/>Backrest Cane Y s/n- *</p> <p>5. Handgrip- TRHG</p> <p>6. Release Handle/Cable Left- STDS4Y4702L<br/>Release Handle/Cable Right- STDS4Y4702R</p> <p>7. Spreader Bar Knob- STDS4Y4715</p> <p>8. Spreader Bar 16"- STDS4Y4720<br/>Spreader Bar 18"- STDS4Y4714<br/>Spreader Bar 20"- STDS4Y4719</p> <p>9. Cable- STDS3006</p> <p>10. Reclining piston Left- STDS4Y4704L<br/>Reclining piston Right- STDS4Y4704R</p> <p>11. Hand Rim Hdwr. Set (9 ea)- STDS004609-9</p> <p>12. Rear Axle Bolt- STDS4Y4707</p> <p>12a. Hub Cap- STDS4Y2431</p> <p>13. Rear Wheel Bearings (pr)- STDS4Y4729</p> <p>14. Anti Tipper- STDS4Y4712</p> <p>15. Rear Wheel- STDS4Y4730</p> <p>16. Caster Bearing- STDS1011</p> <p>17. Caster Axle- STDS3J2431</p> <p>18. Caster- STDS3J2432-1</p> <p>19. Fork (Raised Dust Cap)- STDS3J4533<br/>Fork (Flat Dust Cap)- STDS4S4533</p> <p>20. Elevating Leg Rest Pair- STDS4Y4706<br/>Elevating Leg Rest Left- STDS4Y4706L<br/>Elevating Leg Rest Right- STDS4Y4706R<br/>Calf Pad- STDS4Q2007BLK<br/>Calf Pad Mounting Bracket- STDS8MCPMB</p> | <p>21. Fork Stem Bearing (ea)<br/>Flat Dust Cap- STDS8M4601<br/>Raised Dust Cap- STDS1060</p> <p>22. Dust Cap (Raised)- 95032F260N06<br/>Dust Cap (Flat)- STDS8M4651<br/>Dust Cap Sub (Flat)- STDS1051</p> <p>23. Wheel Lock Right (Long Handle)- **<br/>Wheel Lock Left (Long Handle)- **<br/>Wheel Lock Right- STDS4S4817R ***<br/>Wheel Lock Left- STDS4S4817L ***</p> <p>24. Seat Upholstery 16"- STDS4Y4717<br/>Seat Upholstery Seat 18"- STDS4Y4701<br/>Seat Upholstery Seat 20"- STDS4Y4799</p> <p>25. Upholstery Hardware (16 ea)- STDS2A5418</p> <p>26. Desk Arm Right- STDS4Y4705R<br/>Desk Arm Left- STDS4Y4705L<br/>Full Arm Right- STDS4Y470RF<br/>Full Arm Left- STDS4Y470FL</p> <p>26a. Desk Arm Pad- STDS4Y4705R<br/>Full Arm Pad- STDS4Y4705L</p> <p>27. Side Frame Right- STDS4Y4711<br/>Side Frame Left- STDS4Y4710</p> <p>27a. Hanger Pin (4 ea)- STDS4S1069</p> <p>28. Cross Frame 16"- STDS4Y4721<br/>Cross Frame 18"- STDS4Y4709<br/>Cross Frame 20"- STDS4Y4727</p> <p>29. Rear Slide Tube 16" &amp; 18"- STDS4Y4798<br/>Rear Slide Tube 20"- STDS4Y4798-20<br/>Front Slide Tube 16" &amp; 18"- STDS4Y4798<br/>Front Slide Tube 20"- STDS8M4925</p> <p>30. Flip Lock Arm Release- STDS4Y2432-1</p> <p>31. Cross Frame Plug- STDS4Y4733</p> <p>** Long Handle brake no longer available.<br/>*** Serial # 4S1506.. &amp; Newer, (replaced long handle brake).</p> |
|---|--|

\* For Y s/n if only 1 backrest cane is needed send two STDS4S4716, 1 at no charge. If 2 are needed send two STDS4S4716.