
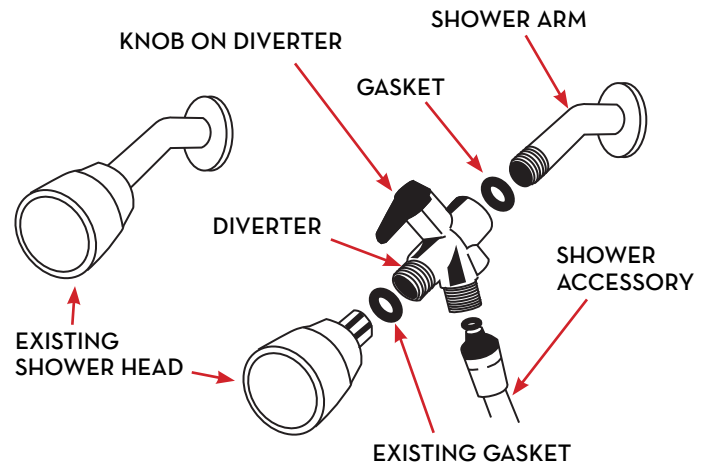
installation instructions (12037 shower spray)

1. Remove existing shower head and gasket from shower arm.
2. Clean the threads on shower arm with steel wool.
3. Apply pipe thread sealing tape onto the shower arm.
4. Place the gasket (provided) into end of diverter valve (Figure A).
5. Using a wrench or pliers, connect large valve to shower arm.
6. Install shower hose onto bottom connection of diverter valve.
7. Re-attach shower head onto front of diverter valve (Figure B)
8. Turn black knob to change from regular shower to shower spray.
9. Turn black knob back to use conventional shower.

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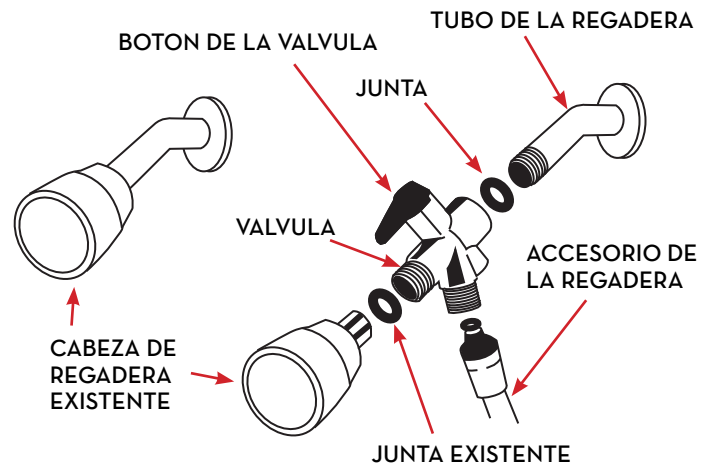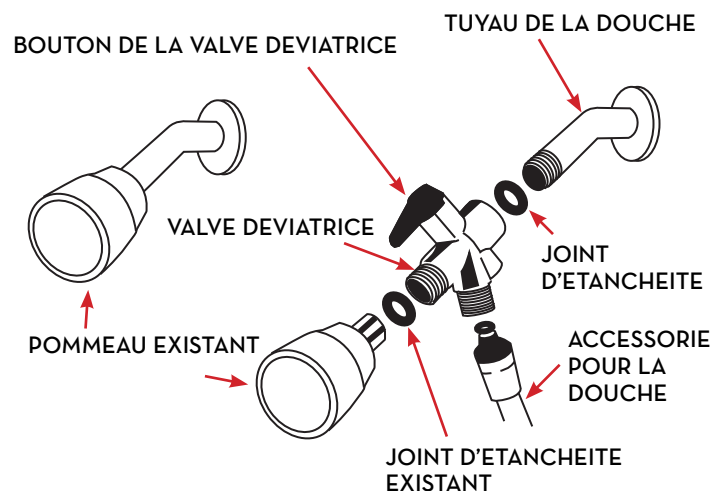
1 year limited warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for one year of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This 1 year Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.

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