


TRIDENT HD

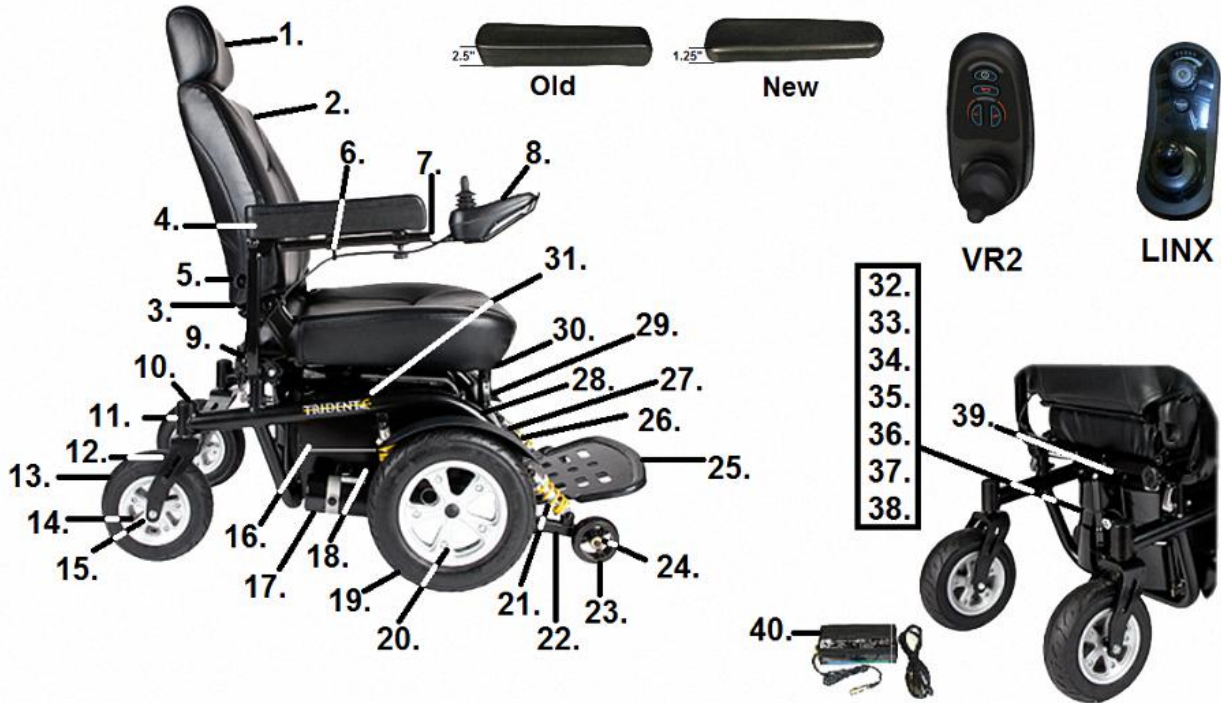

- | | |
|--|--|
| <p>1. Headrest- TRIDHD-26</p> <p>2. 22" Seat- TRIDHD-07
24" Seat- TRIDHD-08
22" Pan Seat Assembly- TRIDHD-33
24" Pan Seat Assembly- TRIDHD-34</p> <p>3. Armrest Assembly Left- TRIDHD-30
Right- TRIDHD-29 ***
*** (w/o Joystick Bracket)
Armrest Assembly Left- TRIDHD-30N ****
Right- TRIDHD-29N ****</p> <p>4. Arm Pad- TRID-2A05
Arm Pad Screws (pr)- TRID-2A51
Arm Pad- TRID-2A05N ****
Arm Pad Screws (pr)- TRID-2A51N ****</p> <p>5. Armrest Height Knob- TRID-2A43</p> <p>6. Joystick Mount- TRID-2A38</p> <p>7. Joystick Arm VR2- TRID-2A36
Linx- TRID-2A36-LINX*</p> <p>8. Joystick VR2- S350169701
Linx- LINX-JOYSTICK*
Joystick Tip Linx- DC GPL55470
Extension Cable- GLM-BUS150-A**</p> <p>9. Armrest Width Knob- TRID-2A44</p> <p>10. Dust Cap- TRIDHD-04</p> <p>11. Fork Stem Bearing- STDS2050</p> <p>12. Fork- TRIDHD-01</p> <p>13. Caster Tire Only- TRIDHD-24</p> <p>14. Caster Complete- TRIDHD-02
Bearings (pr)- TRIDHD-32</p> <p>15. Caster Axle- TRIDHD-03</p> <p>16. Motor Shock Absorber- TRIDHD-23</p> <p>17. Motor/Gearbox Assembly
VR2 Left- TRIDHD-17
VR2 Right- TRIDHD-18
Linx Left- TRIDHD-17-DY*
Linx Right- TRIDHD-18-DY*
Motor Brush (ea) VR2 & Linx- SF5012</p> <p>18. Motor Mount- - TRIDHD-09</p> <p>19. Drive Tire Only- TRIDHD-27</p> <p>20. Drive Wheel Complete- TRIDHD-06
(includes mounting nut & washer)</p> | <p>21. Anti-Tipper Shock Absorber- TRID-2A19</p> <p>22. Anti-Tipper Arm- TRID-2A18</p> <p>23. Anti-Tipper Wheel- TRID-2A16</p> <p>24. Anti-Tipper Axle- TRID-2A17</p> <p>25. Footrest & Bolt- TRIDHD-21</p> <p>26. Footrest Bracket & Knob- TRIDHD-22</p> <p>27. Fender (Left or Right)- - TRIDHD-19</p> <p>28. Main Frame- TRIDHD-10</p> <p>29. Height Adjustment Pins (3 ea)- TRID-2A28</p> <p>30. Upper Frame- TRIDHD-11</p> <p>31. Seat Latch- TRIDHD-20</p> <p>32. Rear Battery Cover- TRIDHD-25</p> <p>33. Battery Case
VR2 Pair- TRIDHD-15
1 Plug- TRIDHD-15-1
2 Plug- - TRIDHD-15-2
Linx Pair- TRIDHD-53-LINX
1 Plug- TRIDHD-15-1
2 Plug- TRIDHD-15-2-LINX</p> <p>34. Battery Tray- TRIDHD-13</p> <p>35. Battery Tray Slides- TRIDHD-12</p> <p>36. Battery Tray Release Pin- TRIDHD-14</p> <p>37. Battery Tray Hold Down Bolt- TRIDHD-16</p> <p>38. Battery (2 needed)- LRB402308</p> <p>39. Controller VR2- TRIDHD-28
Linx- TRIDHD-28-LINX
Battery to Controller Cable- TRIDHD-31</p> <p>40. Charger- CPC
When Stock is gone use LRA402210.</p> <p>ELR Adapter kit
Seat Plate Bracket- TRID-2A47
ELR Bracket- SFELR
ELR- LK3JELR</p> <p>* Ser # 2A160800105 & later
** Linx Joystick Only
**** Ser # 2A1707600001 & later</p> |
|--|--|