[www.drivemedical.com](http://www.drivemedical.com)

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[www.drivemedical.com](http://www.drivemedical.com)

**drive**  
MEDICAL DESIGN & MANUFACTURING

## installation instructions

This bed assist board is designed for use on a bed with a box spring and mattress.

1. Remove contents from carton.
2. Remove mattress from bed.
3. Unfold the top and bottom extension (figure 1) and lie flat on the bed surface.
4. Attach the hand brace (figure 2) to the base support frame by depressing the brass lock pins and sliding the hand brace into the tubing (figure 3) until the lock pins “snap” through the positioning holes.
5. Locate a convenient distance from the head of the bed to position the bed assist rail.
6. Replace mattress on top of box spring.

figure 1

figure 2

figure 3

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figure 3

## lifetime limited warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for the lifetime of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.

© 2006 Medical Depot, Inc. All rights reserved.  
Drive is a trademark of Medical Depot, Inc.  
Port Washington N.Y. 11050 USA Made in China

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© 2006 Medical Depot, Inc. Todos los derechos reservados.
Drive es marca registrada de medical depot, Inc.
© 2006 Medical Depot, Inc. Tous droits réservés.
Drive est une marque de commerce de Medical Depot, Inc.