

two year limited warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for two years from the date of the original consumer purchase.

This device was built to exacting standards and carefully inspected prior to shipment. This Two Year Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of dependable service.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.



*Wenzel*ite*Re/hab*

© 2006 Medical Depot, Inc. All rights reserved.

Drive is a trademark of Medical Depot, Inc.

Port Washington N.Y. 11050 USA Made in China

otter shower stand
item # OT 8020



EU Authorized Representative
Drive International, LLC
Globe Lane - Stockport
DUKINFIELD SK16 4RE, GB

*Wenzel*ite*Re/hab*


www.drivemedical.com

drive
MEDICAL DESIGN & MANUFACTURING

recommended use

The Otter Tub Stand elevates the Otter Bath Chair for use in the shower. All three Otter Bath Chair sizes fit on the Shower Stand. It is constructed of aluminum.


assembly instructions

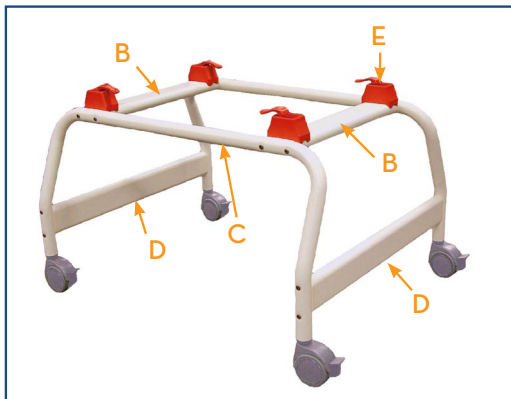

step 1

Insert four plastic end caps (A) into both lever crossbars (B) and connect crossbars to the two side frames (C) using screws supplied. Do not tighten screws.

NOTE: All triggers must face toward the center of base.

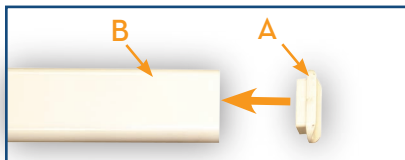

step 2

Insert four plastic end caps (A) into leg crossbars (D) and connect crossbars to the two side frames using screws supplied.

step 3

Tighten all screws.

maintenance and care

1. Inspect the shower stand periodically for cracks, breaks, loose or missing parts or malfunctions. If a product needs service or replacement parts, contact an authorized Drive dealer.
2. Wash shower stand with soap and water or 10% bleach.

adjustments

1. To mount Otter Bath chair onto Shower Stand, position bath chair legs to the highest position and push down firmly on chair until legs snap into sockets (E) of shower stand. To remove chair from Shower Stand, depress red levers and pull legs upward one at a time.
2. Lock all casters to immobilize and unlock for mobility.

CAUTION:

Adult supervision is required at all times.

