super lite
transport chair, 18"

silla de transporte
súper liviana, 18"

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www.drivemedical.com

maintenance tips

Proper care and maintenance is important in order to keep your Super Lite Transport Chair in good working condition. Be sure that your chair is in proper working condition at all times. A qualified dealer or repair center (wheelchair rental dealer) should be consulted for periodic inspections and repairs.

1. At least once a week, wipe off the frame using a clean and soft cloth. If the chair is exposed to moisture, dry the chair immediately. The metal parts of the chair should be replaced promptly as it may not support your body weight.
2. Clean the upholstery with a mild soap solution, rinse and wipe dry. Worn or torn upholstery should be replaced promptly as it may not support your body weight.
3. Inspect the wheels at least once a month. Correct any side play by loosening the lock nut on the axle, then tighten the axle just enough to remove side play.
4. Solid rubber tires require limited upkeep, but need to be cleaned periodically with a damp cloth. Solid rubber tires need to be replaced if they become cracked or severely worn.

It is important for the attendant to be familiar with proper assisting procedures in any situation that requires the use of the Super Lite Transport Chair. Safety is the key consideration. Consult your physician, nurse or physical therapist to determine methods most suitable for your individual needs and abilities.

operating tips

Your Transport Chair Dealer (Wheelchair rental Dealer) can assist you in making adjustments and provide with operational information.

1. To open the chair, pull up on back support in opposite direction of footrest.
2. To fold down and store, stand behind chair, pull release chair towards while pushing the top of the back support away from you.
3. To deploy footrest simply pull down the footrest support. Pull the footrest support up when not in use.

safety tips

1. Before entering or leaving the chair, engage the dual wheel locks against the tire on both rear wheels. Make sure the front casters are in the forward position before transferring into or out of the chair.
2. Do not move forward while leaning forward in the chair.
3. Leaning out of the transport chair without proper assistance could cause tipping.

NOTE: You and your attendant should consult a qualified professional for the correct techniques on ramp negotiations.

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silla de transporte súper liviana

(Item # SL18)

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drive
MEDICAL DESIGN & MANUFACTURING

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lifetime limited warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for the lifetime of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.

© 2006 Medical Depot, Inc. All rights reserved.

Drive is a trademark of Medical Depot, Inc.

Port Washington N.Y. 11050 USA Made in China

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© 2006 Medical Depot, Inc. Todos los derechos reservados.
Drive es marca registrada de medical depot, Inc.
© 2006 Medical Depot, Inc. Tous droits réservés.
Drive est une marque de commerce de Medical Depot, Inc.