[www.drivemedical.com](http://www.drivemedical.com)

drive™

Rev.2.07.23.14

transfer bench with  
sliding seat and fold up legs

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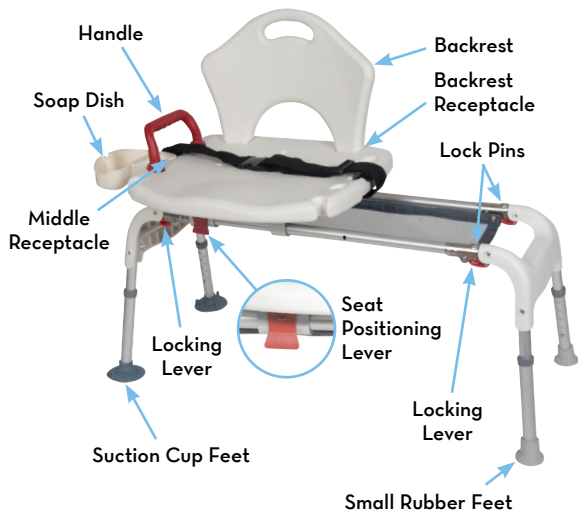
item # RTL12075

EU Authorized Representative  
Drive International, LLC  
Globe Lane - Stockport  
DUKINFIELD SK16 4RE, GB

[www.drivemedical.com](http://www.drivemedical.com)

drive™

## assembly instructions

1. Remove contents from carton.
2. Place transfer bench frame on its side. Unfold one leg section by extending outward. Locking pins (2) will snap into place when legs are secure. Make sure both locking pins fully protrude through the holes.
3. Repeat for second leg section.
4. Stand transfer bench upright. Insert backrest into backrest receptacle on back of the seat. Push down firmly to secure in place.
5. Attach red handle by inserting handle into the two receptacle holes on the side of the seat. Be sure that curve of handle is facing out. Note: handle should be inserted on the same side as the large suction cup feet. Secure in place by inserting the two (2) red locking knobs from the bottom and turning clockwise until tight.
6. Insert soap dish on same side as red handle by sliding into middle receptacle from the top down.

Rev.2.07.23.14

## operation

1. Place transfer bench in tub so the suction cup feet are **INSIDE** the tub. (If the preference is for the seat to face the opposite way, legs can be switched to the other side of the bench by depressing brass push pins, removing legs and inserting them into the opposite leg receptacles.)
2. Adjust height of legs to desired height by depressing brass push pins and setting to proper height. Note: Legs should be set to ensure seat is level, taking into account that the legs inside the tub will have to be set to a lower height setting because of the difference in height from most tubs to floors.
3. To move seat, pull the red seat positioning lever located under the seat away from the frame and slide seat until it clicks into place. Note: there are two (2) positions where seat will be secured to frame, one on the left side and one on the right side.

## folding

To fold for storage or transport, stand bench vertically on one set of legs. On one side of leg assembly, pull both red locking levers away from frame simultaneously until both locking pins disengage. Fold legs down. Turn transfer bench over and repeat for second set of legs. Remove backrest, handle and soap dish if necessary.

### CAUTION:

When entering or exiting shower or bath via the transfer bench, make sure the seat is in one of the locked positions.

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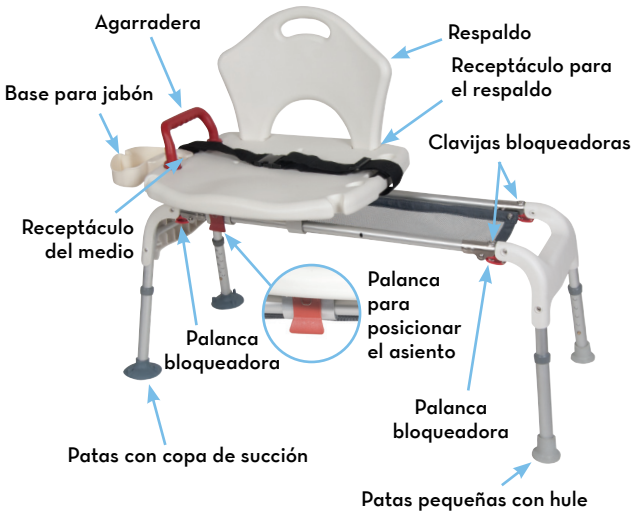
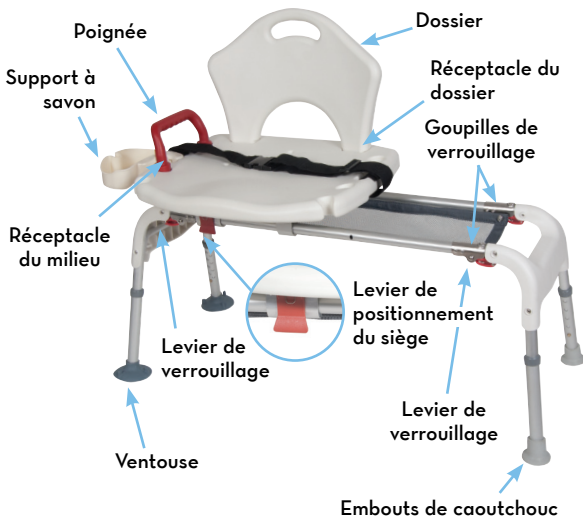
## lifetime limited warranty

Your Drive branded product is warranted to be free of defects in materials and workmanship for the lifetime of the product for the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.

© 2014 Medical Depot, Inc. All rights reserved.  
Drive is a trademark of Medical Depot, Inc.  
Port Washington N.Y. 11050 USA Made in China

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©2014 Medical Depot, Inc. Todos los derechos reservados.
Drive es marca registrada de medical depot, Inc.
© 2014 Medical Depot, Inc. Tous droits réservés.
Drive est une marque de commerce de Medical Depot, Inc.