


## TRIDENT 2A Serial #

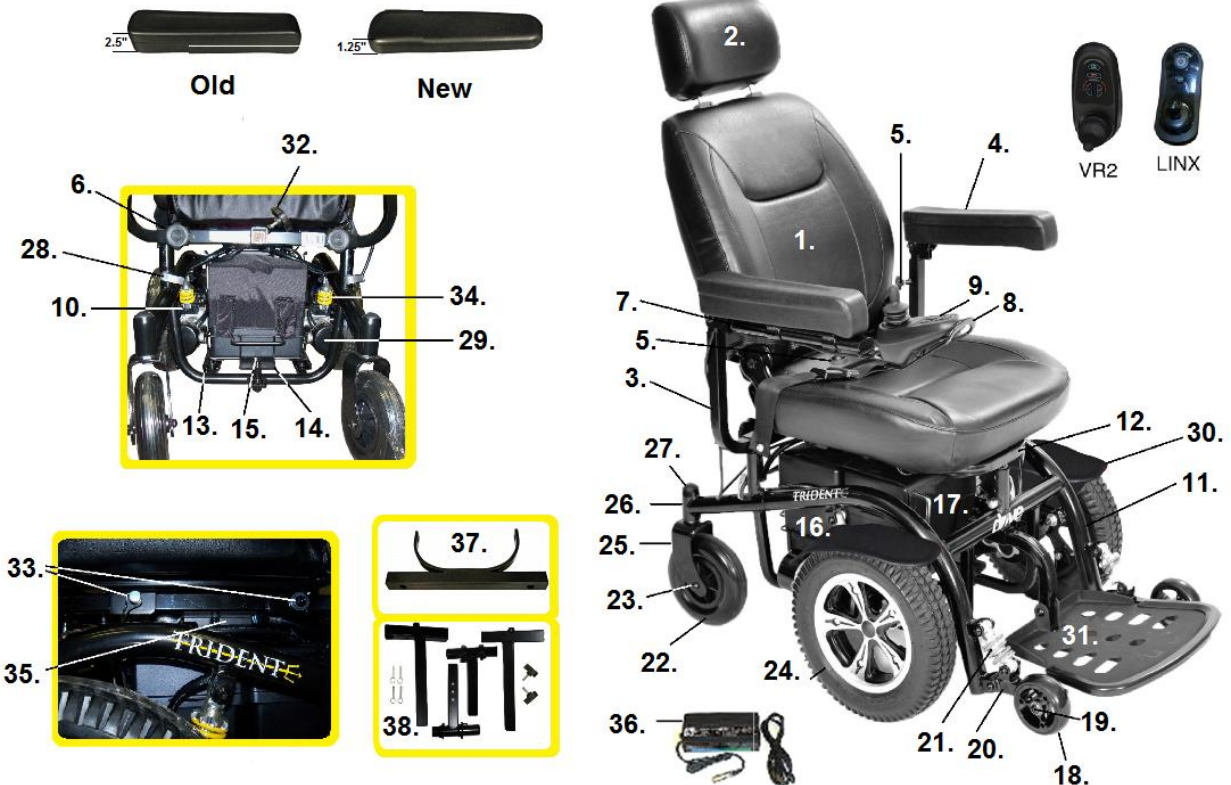

- |   |   |
|---|---|
| <p>1. Seat Assembly w/Headrest 18"- TRID-2A48 **<br/>         Seat Assembly w/Headrest 20"- TRID-2A49 **<br/>         Seat Assembly w/Headrest 18"- TRID-2A01<br/>         Seat Assembly w/Headrest 20"- TRID-2A02</p> <p>2. Headrest- TRID-2A39</p> <p>3. Armrest Assembly Old Left- TRID-2A04<br/>         Armrest Assembly Old Right- TRID-2A03<br/>         Armrest Assembly New Left- TRID-2A04N ***<br/>         Armrest Assembly New Right- TRID-2A03N ***</p> <p>4. Arm Pad- TRID-2A05<br/>         Arm Pad Screws (pr)- TRID-2A51<br/>         Arm Pad- TRID-2A05N ***<br/>         Arm Pad Screws (pr)- TRID-2A51N ***<br/>         Joystick Cable Screws (3 ea)- TRID-2A46</p> <p>5. Knob Joystick or A/R Height- TRID-2A43</p> <p>6. Knob Armrest Width- TRID-2A44</p> <p>7. Joystick Mounting Bracket- TRID-2A38</p> <p>8. Joystick Fixed Arm- TRID-2A36<br/>         Joystick Swing-Away Arm- TRID-2A37<br/>         Joystick Fixed Arm Linx- TRID-2A36-LINX<br/>         Joystick Swing-Away Arm Linx- TRID-2A37-LINX</p> <p>9. Joystick VR2- S350169701<br/>         Joystick Linx- LINX-JOYSTICK<br/>         Joystick Tip Linx- DC GPL55470<br/>         Joystick/Cntrl Cable Linx- GLM-BUS150-A ****</p> <p>10. Joystick Cable Clips (3 ea)- TRID-2A46</p> <p>11. Motor Mount Arm- TRID-2A20</p> <p>12. Main Frame- TRID-2A22</p> <p>13. Upper Frame- TRID-2A23<br/>         Upper Frame- TRID-2A50 **</p> <p>14. Battery Tray Slides (pr)- TRID-2A25</p> <p>15. Battery Tray- TRID-2A24<br/>         Battery Tray Release Pin- TRID-2A26</p> <p>16. Battery Case 1 Plug- TRID-2A32<br/>         Battery Case 2 Plug- TRID-2A33<br/>         Battery Case 1 Plug Linx- TRID-2A54-LINX ****<br/>         Battery Case 2 Plug Linx- TRID-2A54-LINX ****<br/>         Battery Cases (pr) Linx- TRID-2A53-LINX ****<br/>         Battery (2 needed)- LRA402208<br/>         Circuit Breaker- LRA402307</p> <p>17. Battery Box Hold Down Cover- TRID-2A45</p> | <p>18. Anti-Tipper Wheel- TRID-2A16</p> <p>19. Anti-Tipper Axle- TRID-2A17</p> <p>20. Anti-Tipper Articulating Arm- TRID-2A18</p> <p>21. Anti-Tipper Shock Absorber- TRID-2A19</p> <p>22. Rear Caster- TRID-2A13<br/>         Caster Bearing- STDS1011</p> <p>23. Caster Axle- TRID-2A14</p> <p>24. Drive Wheel w/Nut &amp; Washer- TRID-2A15</p> <p>25. Fork- TRID-2A10</p> <p>26. Fork Stem Bearing- TRID-2A11</p> <p>27. Fork Stem Cap- TRID-2A12</p> <p>28. Height Adjustment Pin (3 ea)- TRID-2A28</p> <p>29. Motor &amp; Gear Box VR2 Left- TRID-2A29<br/>         Motor &amp; Gear Box VR2 Right- TRID-2A30<br/>         Motor Brush &amp; Cap (pr)- TRID-2A31<br/>         Motor &amp; Gear Box Left Linx- TRID-2A29-LINX ****<br/>         Mtr &amp; Gear Box Right Linx- TRID-2A30-LINX ****<br/>         Motor Brush Linx (ea)- CP5201 ****<br/>         Motor Brush Cap Linx (ea)- CP5202 ****</p> <p>30. Wheel Guard Lef - TRID-2A41<br/>         Wheel Guard Right- TRID-2A42</p> <p>31. Footrest &amp; Attaching Bolt- TRID-2A07<br/>         Footrest Bolt- TRID-2A08<br/>         Footrest Bracket &amp; Knob- TRID-2A09</p> <p>32. Accessory Knob- TRID-2A44</p> <p>33. Seat Brackets (4 ea)- TRID-2A06</p> <p>34. Motor Mount Shock Absorber- TRID-2A21<br/>         Shock Absorber Hardware- TRID-2A34</p> <p>35. Controller- TRID-2A35<br/>         Controller Linx- TRID-2A35-LINX ****</p> <p>36. Charger- CPC<br/>         When stock is gone use LRA402210</p> <p>37. Seat Plate Adapter Bracket- TRID-2A47 **</p> <p>38. ELR Adapter Bracket- SFELR *</p> |
|---|---|

\* LK3JELR use with 37 & 38.  
 \*\* Serial # 2A1406001520 and later.  
 \*\*\* Serial # 2A1707600001 and later.  
 \*\*\*\* Serial # 2A1608600105 and later.