


CIRRUS PLUS EC

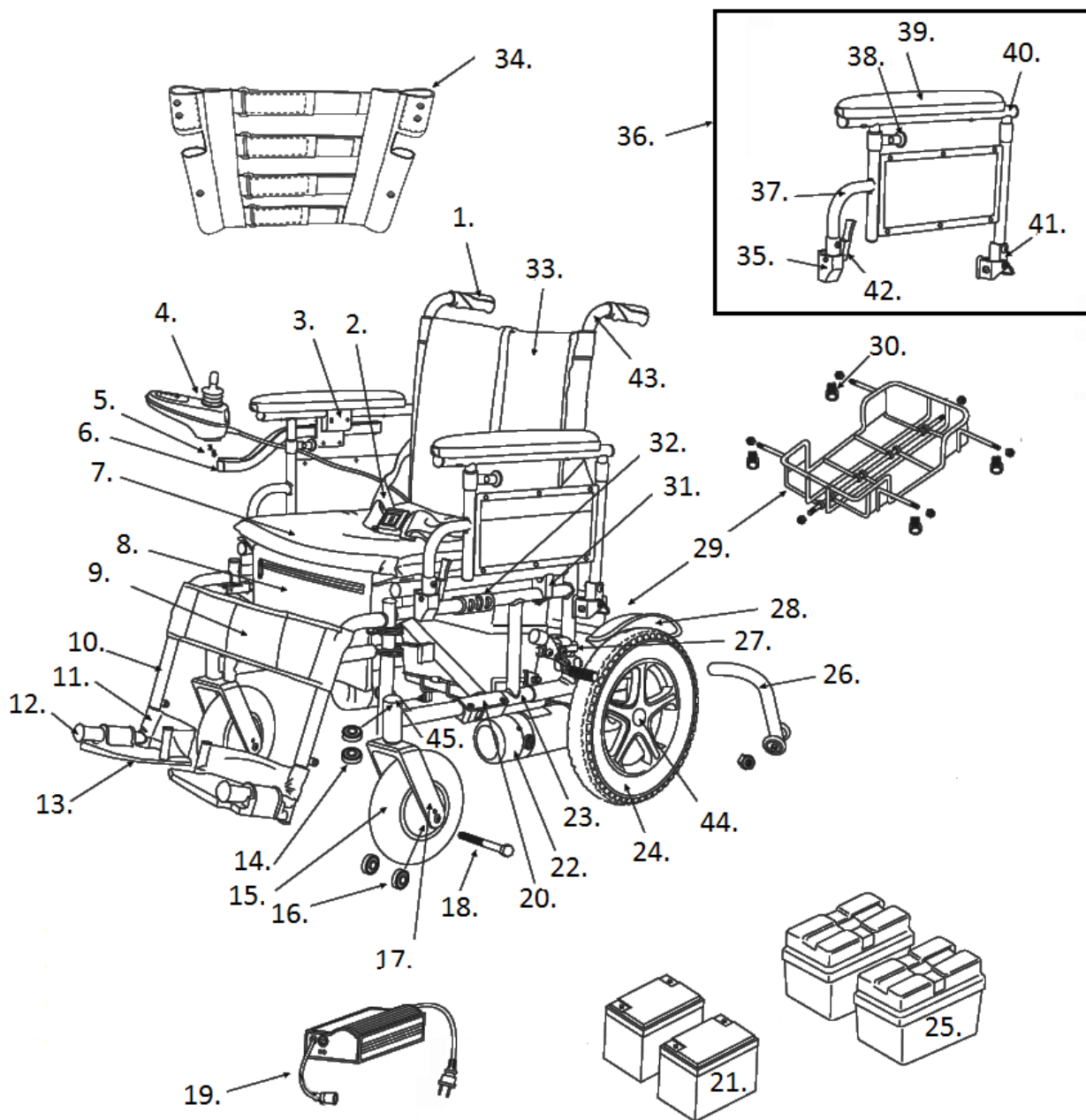

- | | |
|--|---|
| 1. Handgrip- STDS871 | Motor Brush Cap (ea)- CP5202 |
| 2. Seat Belt- STDS850 | 23. Side Frame Left- CPN8017L
Side Frame Right- CPN8017R |
| 3. Arm Clamp*- CPN6205
* Clamp to clamp button head-
screw size: M6X1.00X6mm- | 24. Rear Wheel- CP8012 |
| 4. Joystick- CP6200PN
Joystick Tip- CP6200T
Joystick Over Lay- CP6200PN-BOOT | 25. Battery Case 1 Plug- CP5211-1
Battery Case 2 Plug- CP5211-2
Battery Case Set - CP5115
Battery Case Set with Batteries- CPLUSBATT
Battery Strap- VENT-09 |
| 5. Joystick Screws- SI69001 | 26. Anti-Tipper (ea)- CP8025 |
| 6. Joystick Arm- CP6205P | 27. Brake Left- CP8035L
Brake Right- CP8035R |
| 7. Seat Upholstery 16"- CP8002-16
Seat Upholstery 18"- CP8002-18
Seat Upholstery 20"- CP8002-20
Seat Upholstery 22"- CP8002-22
Seat Upholstery 24"- CP8002-24
Seat Cushion 16"- CP8001-16
Seat Cushion 18"- CP8001-18
Seat Cushion 20"- CP8001-20
Seat Cushion 22"- CP8001-22
Seat Cushion 24"- CP8001-24 | 28. Fender Left- CPN801L
Fender Right- CPN801R |
| 8. Seat Bag- CPN8010 | 29. Folding Basket w/o Clips 16"- CP8015-16
Folding Basket w/o Clips 18"- CP8015-18
Folding Basket w/o Clips 20"- CP8015-20
Folding Basket w/o Clips 22"- CP8015-22
Folding Basket w/o Clips 24"- CP8015-24 |
| 9. Calf Strap 16"- CP6039
Calf Strap 18"- CP8039
Calf Strap 20"- CP2039
Calf Strap 22"- CP2239
Calf Strap 24"- CP2439 | 30. Basket Clip Rear- CP5005C
Basket Clip Front- CP5005CF |
| 10. Footrest Left- CPNSFL
Footrest Right- CPNSFR
Footrest Pair- CPNSF
Elevating Leg Rest Pair- CPNELR | 31. Seat Saddle- CPN8005 |
| 11. Heel Loop (ea)- STDS004048
Heel Loop (pr)- STDS831 | 32. Cross Frame Link 16"- CP8019-16
Cross Frame Link 18"- CP8019-18
Cross Frame Link 20"- CP8019-20
Cross Frame Link 22"- CP8019-22
Cross Frame Link 24"- CP8019-24 |
| 12. Bumper- STDS1026 | 33. Back Upholstery 16"- CP8006-16
Back Upholstery 18"- CP8006-18
Back Upholstery 20"- CP8006-20
Back Upholstery 22"- CP8006-22
Back Upholstery 24"- CP8006-24 |
| 13. Foot Plate Left- CP8041L
Foot Plate Right- CP8041R | 34. Tension Back 16"- CP8005-16
Tension Back 18"- CP8005-18
Tension Back 20"- CP8005-20
Tension Back 22"- CP8005-22
Tension Back 24"- CP8005-24 |
| 14. Fork Stem Bearing (ea)- STDS1060 | 35. Front Seat Saddle/Arm Receiver Lt.- CPN8008L
Front Seat Saddle/Arm Receiver Rt.- CPN8008R |
| 15. Caster- CP8040 | 36. Armrest Assembly Left- CPN800L
Armrest Assembly Right- CPN800R |
| 16. Axle Bearing (ea)- STDS1011 | 37. Lower Arm Frame- CPN8014 |
| 17. Fork - CP1204 | 38. Adjustment Lock- CPN8009 |
| 18. Caster Axle- CP1204-A | 39. Full Arm Pad- STDSFLNB |
| 19. Charger- CPC
When stock is gone use:- LRA402210 | 40. Upper Arm Frame- CPN8004 |
| 20. Cross Frame 16"- CP8020-16
Cross Frame 18"- CP8020-18
Cross Frame 20"- CP8020-20
Cross Frame 22"- CP8020-22
Cross Frame 24"- CP8020-24 | 41. Pivot Bracket- CPN8006 |
| 21. Battery (ea) (2 needed)- LRA402208 | 42. Release Lever- CPN8007 |
| 22. Motor Assembly Left- CP8050L
Motor Assembly Right- CP8050R
Motor Brush (ea)- CP5201 | 43. Back Cane- CPN8017
Back Cane Adjustment Knob- STDS895 |
| | 44. Hub Cap- CPN8021 |
| | 45. Dust Cap- STDS1051 |