


SENTRA EC (Y, 1S & 4S SERIAL #)

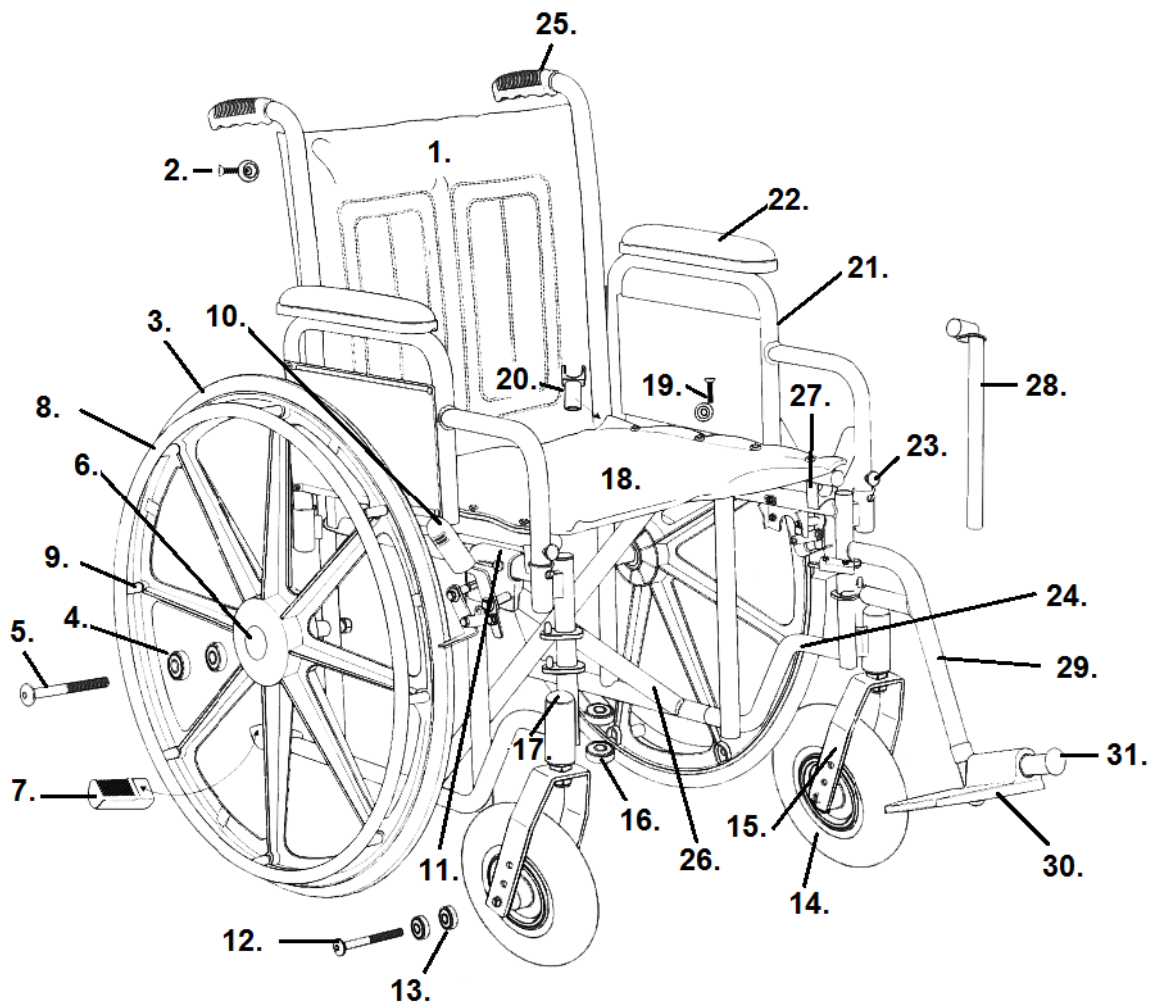

- | | |
|---|---|
| <ul style="list-style-type: none"> 1. Back Upholstery 20"- STDS1S2630
Back Upholstery 22"- STDS1S2632
Back Upholstery 24"- STDS1S2634 2. Back Upholstery Hardware (8 ea)- STDS5D5504 3. Rear Wheel- STDS1S1000HD 4. Rear Axle Bearing (ea)- STDS1S1012 5. Rear Axle- STDS1S1053 6. Hub Cap- STDS1055 7. Tipping Sleeve- STDS4020 8. Hand Rim- STDS004010 9. Hand Rim Hardware (8 ea)- STDS1016-8 10. Wheel Lock Tip- STDS8M4603 11. Wheel Lock Left- STDS4019L
Wheel Lock Right- STDS4019R 12. Front Axle- STDS8M4631 13. Front Axle Bearing (ea)- STDS1011HD 14. Caster- STDS1075-2 15. Fork- STDS1S3045 16. Fork Stem Bearing (ea)- STDS8M4601 17. Dust Cap- STDS1051 18. Seat Upholstery 20"- STDS1S2650
Seat Upholstery 22"- STDS1S2652
Seat Upholstery 24"- STDS1S2654 19. Seat Upholstery Hardware (8 ea)- STDS1072-8 20. Seat Rail Guide- STDS1070 | <ul style="list-style-type: none"> 21. Armrest Assembly Desk Left- STDSDDAECL
Armrest Assembly Desk Right- STDSDDAECL
Armrest Assembly Full Left- STDSDFAECL
Armrest Assembly Full Right- STDSDFAECL
A/R Assy. Adj. Ht. Desk Left- STDSADDASPL
A/R Assy. Adj. Ht. Desk Right- STDSADDASPR
Armrest Assy. Adj. Ht. Full Left- STDSADFASPL
Armrest Assy. Adj. Ht. Full Right- STDSADFASPR 22. Arm Pad Desk- STDSDSKVB
Arm Pad Full- STDSFLVB 23. Arm Release Button- STDSSB 24. Side Frame Left- STDS1S262L
Side Frame Right- STDS1S262R
Side Frame Left- STDS4S262L-N *
Side Frame Right- STDS4S262R-N * 25. Hand grip- STDS871 26. Cross Frame 20"- STDSY2620
Cross Frame 22"- STDSY2622
Cross Frame 24"- STDSY2624 27. Seat Saddle- STDS1M2218 28. Slide Tube- STDS005200 29. Footrest (pr)- STDSF-TF 30. Foot Plate- STDS00004A 31. Footrest Bumper- STDS1026 <p>* 20" side frame after serial # 4S1608153724
* 22" side frame after serial # 4S1609111400
* 24" side frame after serial # 4S1610141127</p> |
|---|---|