


SENTRA HD XW

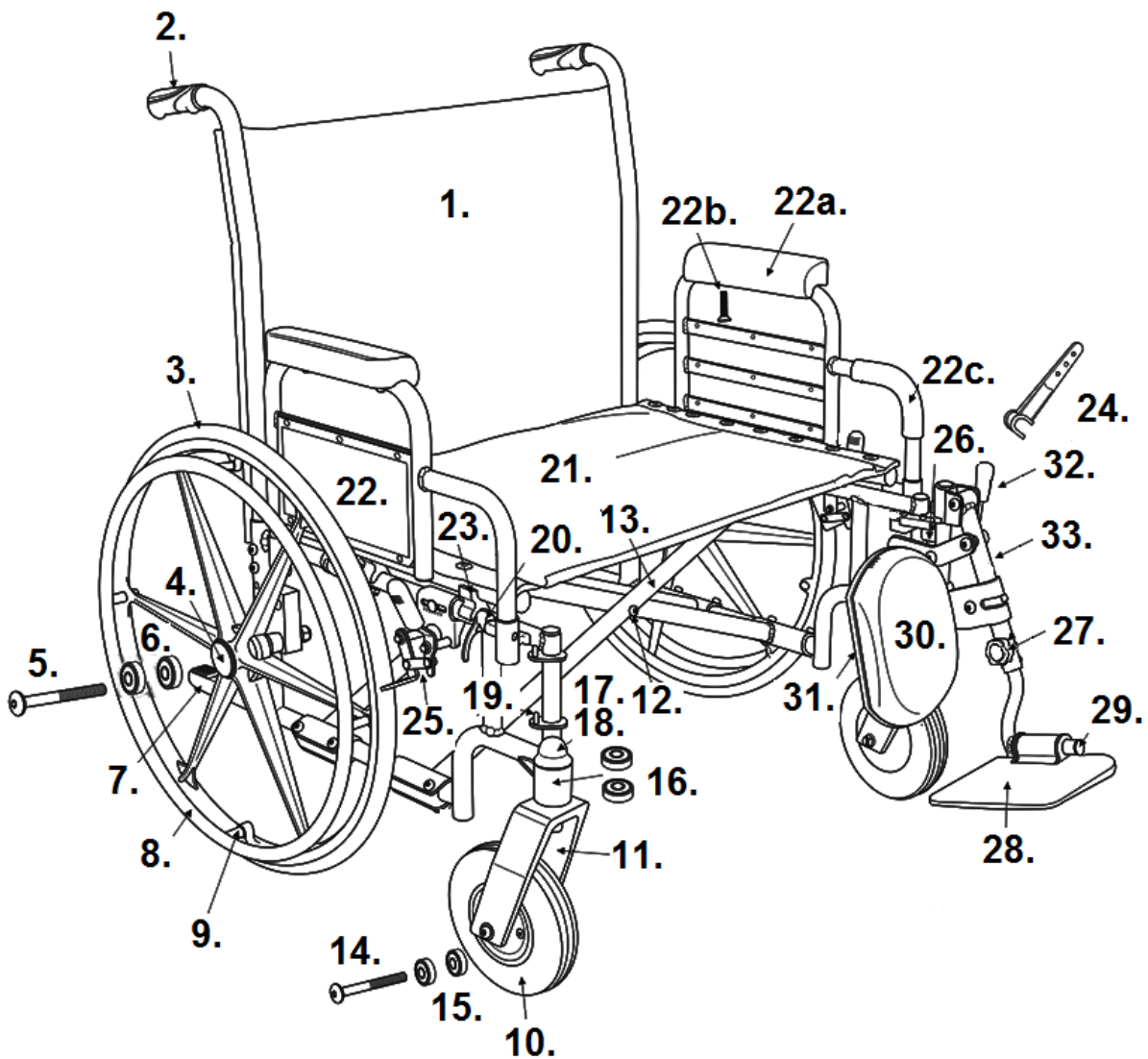

- | | |
|--|--|
| <p>1. Back Upholstery 26"- STDS2389BLK
Back Upholstery 28"- STDS2391BLK
Back Upholstery 30"- STDS2393BLK</p> <p>2. Handgrip- STDS871</p> <p>3. Wheel Assembly- STDS1076</p> <p>4. Rear Bearing Hub Cap- STDS2A5301</p> <p>5. Rear Axle Bolt & Nut- STDS2A5302</p> <p>6. Rear Wheel Bearing- STDS2A5303</p> <p>7. Tipping Sleeve- STDS4020</p> <p>8. Hand Rim- STDS2A5310</p> <p>9. Hand Rim Hardware (ea)- STDS2A5311</p> <p>10. Caster & Fork- STDS1077
Caster Only- STDS2A5320</p> <p>11. Fork Only- STDS2A5321</p> <p>12. Cross Frame Hinge Bolt- STDS001061</p> <p>13. Cross Frame 26"- STDS2A5326
Cross Frame 28"- STDS2A5328
Cross Frame 28"- *</p> <p>14. Front Axle Bolt & Nut- STDS2A5304</p> <p>15. Front Caster Bearings- STDS002050</p> <p>16. Fork Stem Bearings- STDS2050</p> <p>17. Lock nut for Fork- STDS2A5316</p> <p>18. Dust Cap for Fork- STDS2A5323</p> <p>19. Hanger Pins (4 ea)- STDS1069-4</p> <p>20. Arm Release Lever- STDS2A5337</p> <p>21. Seat Upholstery 26"- STDS2388BLK
Seat Upholstery 28"- STDS2390BLK
Seat Upholstery 30"- STDS2392BLK</p> | <p>22. Desk Arm Assembly Left- STDSHDDDAL
Desk Arm Assembly Right - STDSHDDAR
Full Arm Assembly Left - STDSHDDFAL
Full Arm Assembly Right - STDSHDDFAR</p> <p>22a. Arm Pad Desk Length- STDS2A5333
Arm Pad Full Length- STDS2A5331</p> <p>22b. Arm Rest Pad Bolts- STDS2A5334</p> <p>22c. Arm Rest Pad Foam- STDS2A5336</p> <p>23. Seat Rail Saddle- STDS4058</p> <p>24. Cross Frame Assist 26"- STDS2A5306
Cross Frame Assist 28"- STDS2A5308
Cross Frame Assist 30"- STDS2A5350</p> <p>25. Wheel Lock Left- STDS4032L
Wheel Lock Right- STDS4032R</p> <p>25a. Wheel Lock Tip- STDS4011</p> <p>26. Frame Bumper- STDS000026</p> <p>27. ELR Height Adjustment Bolt- HDELR-KNOB</p> <p>28. Footplate Left- STDS2A5353
Footplate Right- STDS2A5351
Footplate and Heel Loop Left- HDELR-FPL
Footplate and Heel Loop Right- HDELR-FPR</p> <p>29. Footrest Bumper- STDS1026</p> <p>30. Calf Pad & Screws- HDELR-CP</p> <p>31. Calf Pad Mtg. Plate & Screw- HDELR-MP</p> <p>32. Adjustment Lever Left- STDS2A536L
Adjustment Lever Right- STDS2A536R</p> <p>33. Elevating Leg Rest (pr)- HDELR</p> <p>* No longer available.</p> |
|--|--|