


## SENTRA HEAVY DUTY (2A SERIAL #)

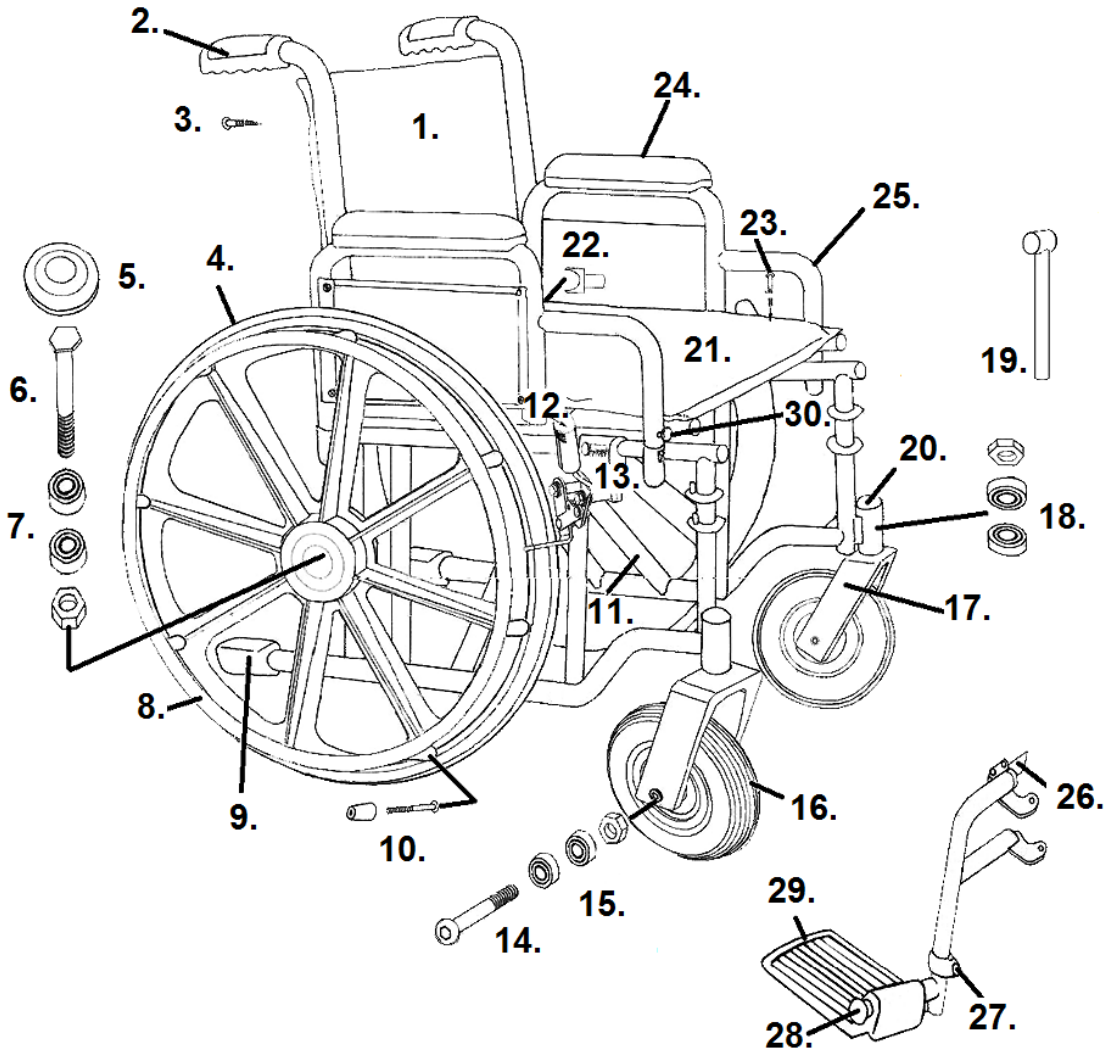

- |  |   |
|--|---|
| <p>1. Back Upholstery 20"- STDS2329BLK<br/>Back Upholstery 22"- STDS2331BLK<br/>Back Upholstery 24"- STDS2333BLK</p> <p>2. Handgrip- STDS871</p> <p>3. Back Upholstery Hardware (8 ea)- STDS5D5504</p> <p>4. Rear Wheel Assembly- STDS1000HD</p> <p>5. Hub Cap- STDS1000C</p> <p>6. Rear Axle- STDS1052</p> <p>7. Rear Wheel Bearing- STDS1012HD</p> <p>8. Hand Rim- STDS005008</p> <p>9. Tipping Sleeve- STDS4020</p> <p>10 Hand Rim Hardware (8 ea)- STDS1016-8</p> <p>11. Cross Frame 22"- STDS2A5222<br/>Cross Frame Bolt- STDS001061</p> <p>12. Wheel Lock Tip- STDS4011</p> <p>13. Wheel Lock Left- STDS4010L<br/>Wheel Lock Right- STDS4010R</p> <p>14. Caster Axle- STDS1014</p> <p>15. Caster Bearing- STDS1011</p> <p>16. Caster- STDS1075<br/>Caster for Hemi Height- STDS1074</p> <p>17. Fork- STDS3045-2</p> <p>18. Fork Stem Bearing- STDS1060</p> <p>19. Slide Tube- STDS005200</p> <p>20. Dust Cap- STDS1051</p> | <p>21. Seat Uphol. 5 Screw Holes 20"- STDS2328BLK<br/>Seat Uphol. 5 Screw Holes 22"- STDS2330BLK<br/>Seat Uphol. 5 Screw Holes 24"- STDS2332BLK<br/>Seat Uphol. 6 Screw Holes 20"- STDS2A2328BLK<br/>Seat Uphol. 6 Screw Holes 22"- STDS2A2330BLK<br/>Seat Uphol. 6 Screw Holes 24- STDS2A2332BLK</p> <p>22. Seat Rail Guide- STDS1070</p> <p>23. Seat Upholstery Hardware (8 ea)- STDS1072-8</p> <p>24. Desk Arm Pad- STDSDSKVB<br/>Full Arm Pad- STDSFLVB<br/>Arm Pad Bolt- STDS002001</p> <p>25. Arm Rest Desk Left- STDSDDASL<br/>Arm Rest Desk Right- STDSDDASR<br/>Arm Rest Full Left- STDSDFASL<br/>Arm Rest Full Right- STDSDFASR<br/>Adjustable Height Desk Left- STDSADDASPL<br/>Adjustable Height Desk Right- STDSADDASPR<br/>Adjustable Height Desk Left- STDSADFASPL<br/>Adjustable Height Desk Right- STDSADFASPR</p> <p>26. Footrest Release Latch Left- STDS00004L<br/>Footrest Release Latch Right- STDS00004R</p> <p>27. Footrest Adjustment Bolt- STDS001051</p> <p>28. Footrest Bumper- STDS1026</p> <p>29. Footplate- STDS00004A</p> <p>30. Arm Release Button- STDSSB</p> |
|--|---|

Footrest- STDSSF-TF

Elevating Leg Rest- STDELRTF

**Side Frame, 20" & 24" Cross Frame are no longer available.**