


**700-830**  
**AIRGO COMFORT-PLUS XC PREMIUM LIGHT WEIGHT TRANSPORT CHAIR**  
**19" WIDE, BLACK**

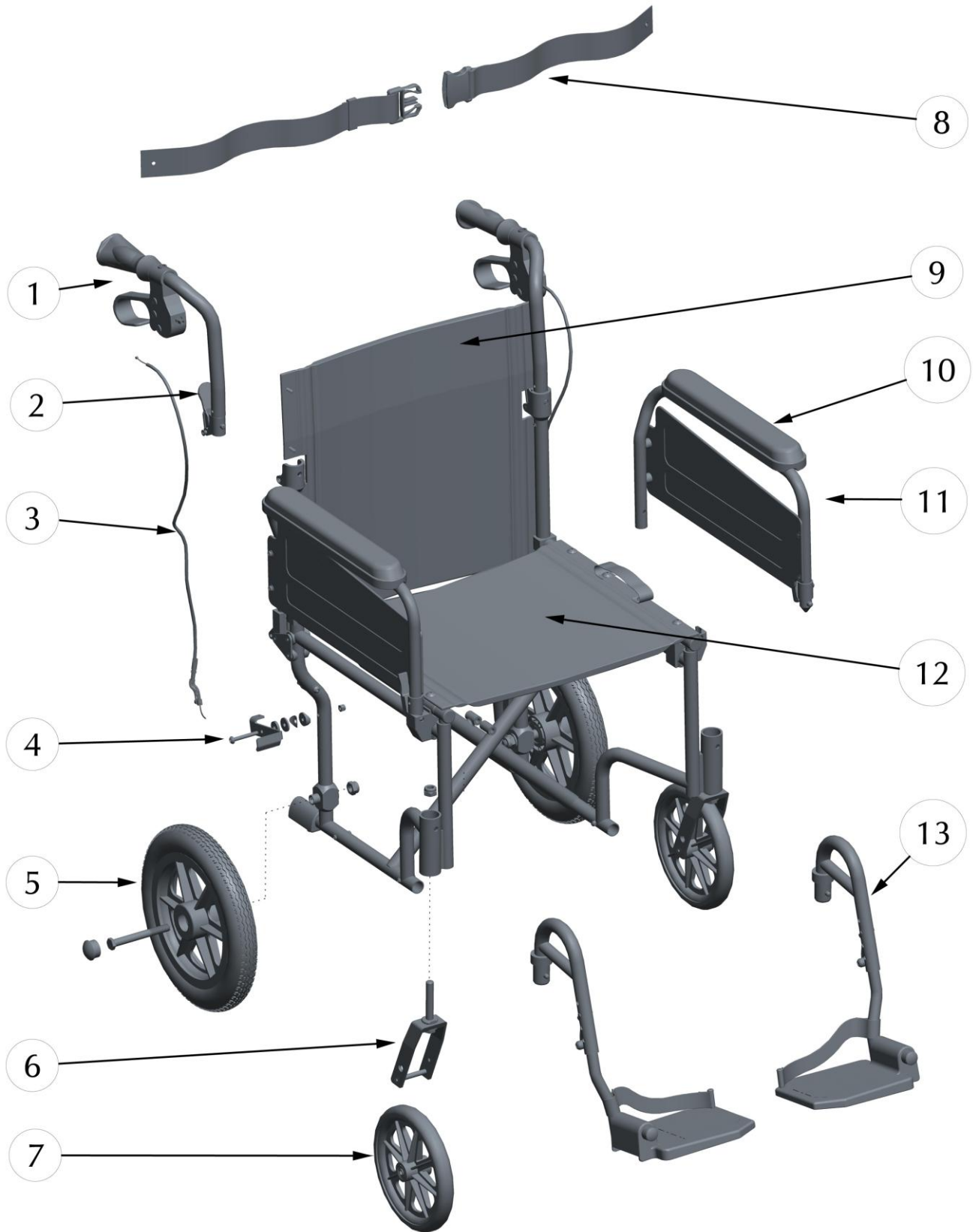

**700-830**  
**AIRGO COMFORT-PLUS XC PREMIUM LIGHT WEIGHT TRANSPORT CHAIR**  
**19" WIDE, BLACK**

Item #	AMG Part #	Item Description	Qty Per Unit	Sold by	UOM
1.	<b>C23-00056</b>	Airgo Handle with Handbrake and Quick Folding Mechanism, L&R	1	1	PR
2.	<b>A02-411</b>	Release Lever to Fold Handle, Black	2	1	EA
3.	<b>C23-00043</b>	Cable, Sleeve and Hardware (Adjustment Screw, Nut, Spring, Cable Stopper & Hex Screw) – 2 ea / set	1	1	ST
4.	<b>C23-00044</b>	Bottom Brake Assy (Bolt M6x45mm, 2 Washers, Curved Plastic Washer, Left & Right Brake Pad, Curved Steel Washer & Hex Nut) -- L & R	1	1	PR
5.	<b>C23-00045</b>	12" Black PU Tire Wheel with 6 Spokes with Hardware (2 Bearings, Hex Bolt M12x96 & Hex Nut)	2	1	EA
6.	<b>C23-00046</b>	Front Aluminum Fork with Hex Flat Bolt M8x58, L&R	1	1	PR
7.	<b>B02-131BK</b>	8" Black PU Wheel	2	1	EA
8.	<b>700-864</b>	Safety Belt for Transport Chair	1	1	EA
9.	<b>B02-463</b>	Back Upholstery 19"x17", Black Nylon, with Hardware (10 Self-threading Screws & 10 Washers)	1	1	EA
10.	<b>A02-461N</b>	Full Arm Pad, Black Nylon – L&R	1	1	PR
11.	<b>C23-00047</b>	Flip-Back, Fixed Height, Detachable Nylon Full Armrest, L&R	1	1	PR
12.	<b>B02-413N</b>	Seat Upholstery 19"x16", Black Nylon, with Hardware (8 Hex Screws & 8 Washers)	1	1	EA
13.	<b>C23-00048</b>	Swing-Away Footrests, Tool-Free Height Adjustable (Aluminum Frame with Composite Footplate & Heel Loops), L&R	1	1	PR