www.drivemedical.com

drive
WWW.DRIVEMEDICAL.COM

bed assist handle

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drive
WWW.DRIVEMEDICAL.COM

installation instructions

This Bed Assist Handle is designed for use with most mattress and box spring type beds.

1. Remove contents from carton.
2. Remove mattress from bed.
3. Attach the hand brace (figure 1) to the base support frame by depressing the brass lock pins and sliding the hand brace into the tubing (figure 2) until the lock pins “snap” through the positioning holes.

figure 1

figure 2

4. Place assembled base support frame on top of the box spring (Figure 3).
5. Unbuckle the strap and lay flat.

figure 3

6. Position the bed assist handle so that the space between the vertical handle bars and other bedside device or bed board must be less than 2.36 inches (60mm), or larger than 9.25 inch (235mm).
7. Place bed assist handle so that the strap is exposed on both sides of the bed (Figure 4). Place mattress on bed.
8. Buckle the strap and adjust by tightening the loose end of the strap (Figure 5).

figure 4

figure 5

WARNING

- This assist handle is intended for assisting in and out of bed only. **DO NOT** use for other purposes.
- Mattress must fit bed frame and assist handle snugly to prevent potential patient entrapment.
- Follow installation instruction to prevent potential patient entrapment.
- Patient entrapment may cause injury or death!

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lifetime limited warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for the lifetime of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.

© 2006 Medical Depot, Inc. All rights reserved.
Drive is a trademark of Medical Depot, Inc.
Port Washington N.Y. 11050 USA Made in China

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© 2006 Medical Depot, Inc. Todos los derechos reservados.
Drive es marca registrada de medical depot, Inc.
© 2006 Medical Depot, Inc. Tous droits réservés.
Drive est une marque de commerce de Medical Depot, Inc.