

## Nebulizer Kit



### **Cleaning of nebulizer:**

After each use, unscrew nebulizer into parts, wash in warm soapy water, and rinse well. Leave it in a clean area to air dry.

### **NOTE:**

Do not towel dry nebulizer parts; this could cause contamination.

### **Cleaning accessories:**

After each use rinse and dry mouthpiece. Never immerse tubing in water. Single patient use for one month.



**drive**<sup>TM</sup>

# Kit de Nebulizador

## **Limpieza Del Nebulizador:**

Después de cada uso, desatornille el nebulizador en partes, lave en agua tibia jabonosa y enjuague bien. Déjelo en un área limpia para secar al aire libre.

**NOTA:** No seque las partes del nebulizador con una toalla; esto puede causar la contaminación.

## **Limpiando Los Accesorios:**

Después de cada uso, enjuague y seque el pedazo de boca. Nunca sumerge la tubería en agua. Uso de un mes para un paciente.

---

# Kit de Nébuliseur

## **Nettoyage du Nébuliseur:**

Après chaque utilisation, dévisser les parties du nébuliseur, nettoyer dans de l'eau chaude et savonneuse et bien rincer. Laisser sécher à l'air libre dans un endroit propre.

**REMARQUE:** Ne pas sécher les pièces du nébuliseur avec une serviette; cela pourrait causer une contamination.

## **Nettoyage des Accessoires:**

Après chaque utilisation, rincer et sécher l'embout buccal. Ne jamais immerger le tuyau dans l'eau. À utiliser pour traitement individuel d'un mois.