


TITAN

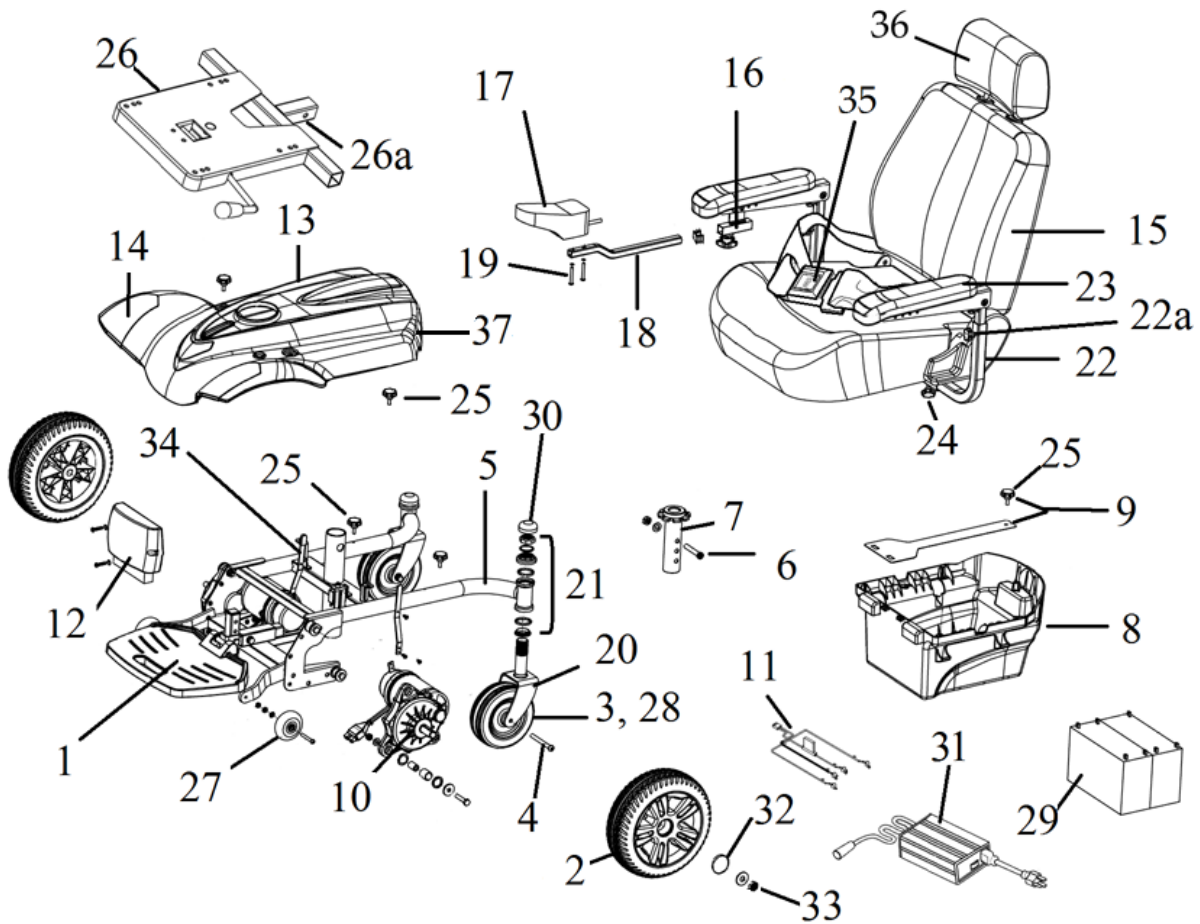

- | | |
|--|--|
| 1. Footplate & Attaching Bracket- TITAN-01
Footrest Depth Adj. Bolt- TITAN-14 | 18. Joystick Arm- LDM400001-04 |
| 2. Drive Wheel Assembly- TITAN-11 | 19. Joystick Mount Screws (pr)- LDM400001-01 |
| 3. Caster Assembly- TITAN-03 | 20. Caster Fork- LDM400013 |
| 4. Caster Axle- TITAN-04 | 21. Fork Stem Bearing- LRB402348 |
| 5. Frame- TITAN-05 | 22. Armrest Assembly Left- LDR410011L
Armrest Assembly Right- LDR410011R |
| 6. Seat Post Bolt- TITAN-06 | 22a. Armrest Height Knob- LDR410011-12 |
| 7. Seat Post (1L Serial #-) TITAN-38 | 23. Arm Pad- LDR410110
Arm Pad Screws (pr)- LDR420018 |
| 8. Battery Tray Assembly- TITAN-07 | 24. Armrest Locking Knob-
1L Serial #- LDR410011-11
VW Serial #- S31052 |
| 9. Battery Hold Down Strap- TITAN-08 | 25. Battery Tray/Shroud Knob-
1L Serial #- LDR410115-03
VW Serial #- VENT-86 |
| 10. Motor & Gear Box Left- TITAN-10
Motor & Gear Box Right- TITAN-09
Axle Key- TITAN-40 | 26. Seat Plate (1L Serial #-) LRC402414
Seat Plate (VW Serial #-) C07-072-01001 |
| 11. Battery Harness- TITAN-12
Circuit Breaker- LRA402307 | 26a. Seat Plate Accessory Screw- LRC402414-02 |
| 12. Controller Shark- TITAN-13
Control/Shroud Cable Shark- LPW182001-03
Controller Linx- DLX-PM40-A-TITAN | 27. Anti-Tipper Wheel (ea)-
1L Serial #- LDR420009-01
VW Serial #- TITAN-41 |
| 13. Shroud- TITAN-16 | 28. Caster Bearing- STDS1011 |
| 14. Decoration Set Red- TITAN-17
Decoration Set Blue- TITAN-18 | 29. Battery (ea) (2 needed)- LRA402208 |
| 15. Captain's Seat Assembly-
16"X16"X18"- TITAN-29
16"X18"X18"- TITAN-30
18"X16"X18"- TITAN-31
18"X18"X16.5"- TITAN-19
18"X18"X18"- TITAN-20
20"X18"X18"- TITAN-21
20"X20"X18"- TITAN-32
20"X20"X19.5"- TITAN-33
Pan Seat 18"- ST501-181818
Pan Seat 20"- ST502-202020 | 30. Dust Cap- LDR4110 |
| 16. Joystick Bracket Knob- LDM400001-07
Joystick Bracket - TITAN-24
Joystick Bracket Sub.- LDM400001-05 | 31. Charger 3 amp- LRA402209 |
| 17. Joystick Shark- LPW182001
Joystick Overlay Shark- DK-REMD01
Joystick/Shroud Cable Shark- LPW182001-02
Joystick Tip Shark- DC GJS0090
Joystick Boot Shark- DC GJS0091
Joystick Linx- LINX-JOYSTICK*
Joystick/Control Cable Linx- GLM-BUS150-A
Joystick Tip Linx- DC GPL55470 | 32. Hub Cap- LRM412202-HC |
| | 33. Drive Wheel Nut & Washer- TITAN-28 |
| | 34. Clutch Handle Assembly- TITAN-26
Clutch Handle Tip- TITAN-27 |
| | 35. Seat Belt- STDS855-PW |
| | 36. Headrest- ST-HEADREST |
| | 37. Rear Reflectors (pr)- TITAN-37 |
| | Not Shown-
ELR Adapter- AE2500
ELR- LK3JELR |

* Serial # VW017398 & higher or (21)2W1704202883 & higher.