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overdoor exercise pulley

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item # 13005

drive
MEDICAL DESIGN & MANUFACTURING

www.drivemedical.com

installation instructions

1. Open box and verify ALL pieces are included.
 - Door Bracket
 - Arm with pulley
 - Handles
 - Exercise cord
2. Holding bracket in one hand, insert top end of arm into notch on the door bracket. Squeezing together the arm, insert bottom of the arm into hole on the door bracket. Securely tie exercise cord to the middle of handle. Run end of exercise cord through **BOTH** pulleys and secure in middle of other handle. Place door bracket securely on top of door unit is to be used. Follow Therapist or Doctor's instructions on use of exercise pulley.

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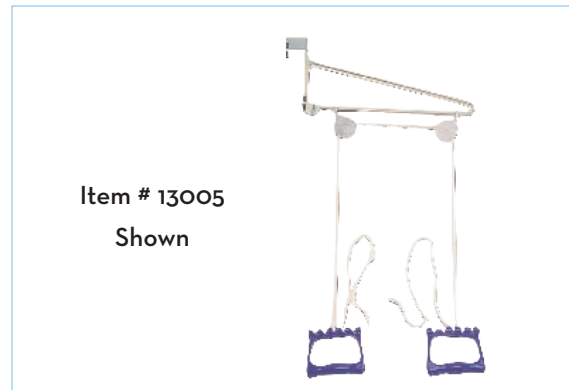
lifetime limited warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for the lifetime of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.

drive
MEDICAL DESIGN & MANUFACTURING

© 2006 Medical Depot, Inc. All rights reserved.
Drive is a trademark of Medical Depot, Inc.
Port Washington N.Y. 11050 USA Made in China