


CIRRUS PLUS EC

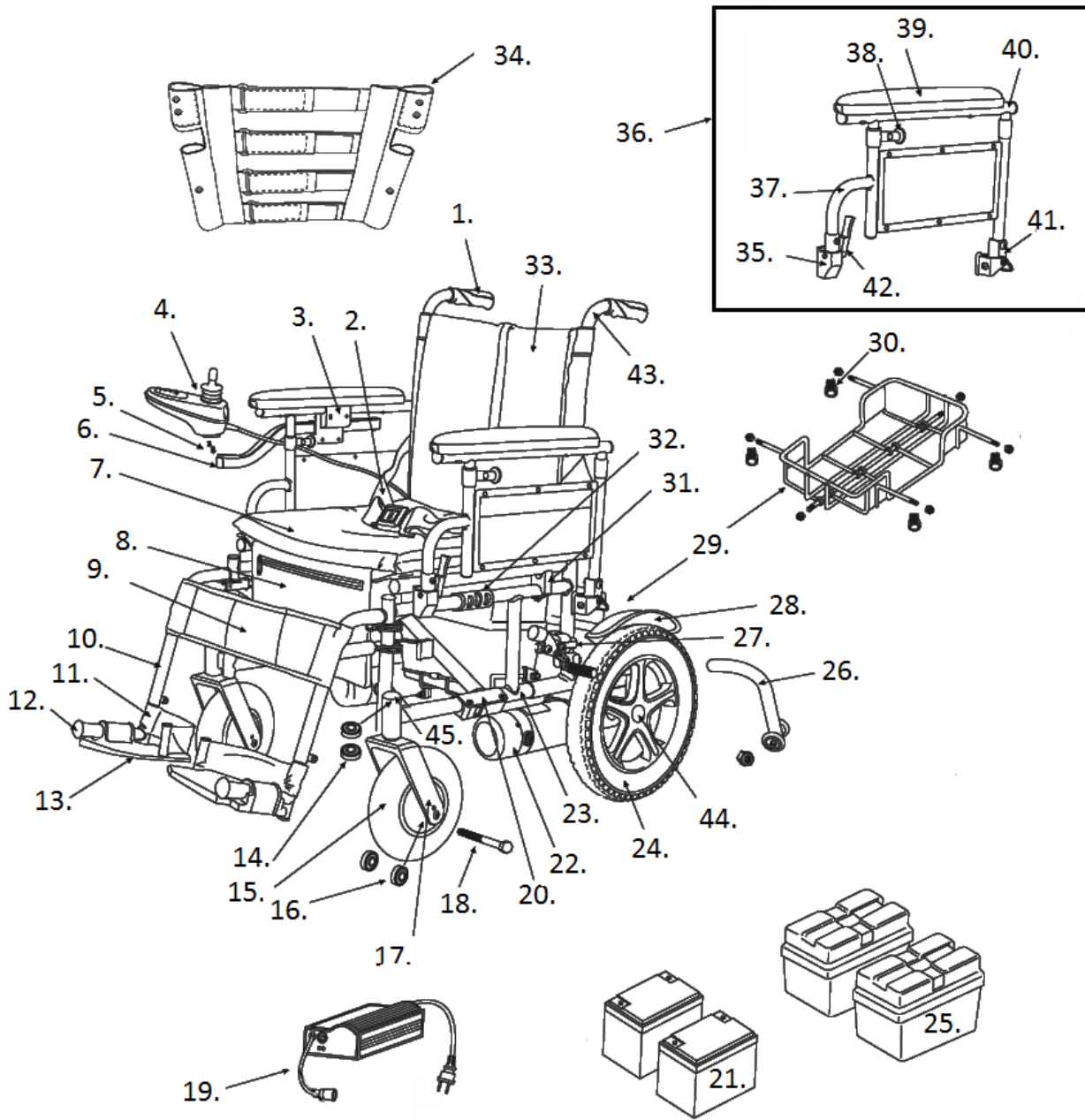

- | | |
|---|---|
| <p>1. Handgrip- STDS871
 2. Seat Belt- STDS850
 3. Arm Clamp*- CPN6205
 * Clamp to clamp button head-
 screw size: M6X1.00X6mm-
 4. Joystick- CP6200PN
 Joystick Tip- CP6200T
 Joystick Over Lay- CP6200PN-BOOT
 5. Joystick Screws- SI69001
 6. Joystick Arm- CP6205P
 7. Seat Upholstery 16"- CP8002-16
 Seat Upholstery 18"- CP8002-18
 Seat Upholstery 20"- CP8002-20
 Seat Upholstery 22"- CP8002-22
 Seat Upholstery 24"- CP8002-24
 Seat Cushion 16"- CP8001-16
 Seat Cushion 18"- CP8001-18
 Seat Cushion 20"- CP8001-20
 Seat Cushion 22"- CP8001-22
 Seat Cushion 24"- CP8001-24
 8. Seat Bag- CPN8010
 9. Calf Strap 16"- CP6039
 Calf Strap 18"- CP8039
 Calf Strap 20"- CP2039
 Calf Strap 22"- CP2239
 Calf Strap 24"- CP2439
 10. Footrest Left- CPNSFL
 Footrest Right- CPNSFR
 Footrest Pair- CPNSF
 Elevating Leg Rest Pair- CPNELR
 11. Heel Loop (ea)- STDS004048
 Heel Loop (pr)- STDS831
 12. Bumper- STDS1026
 13. Foot Plate Left- CP8041L
 Foot Plate Right- CP8041R
 14. Fork Stem Bearing (ea)- STDS1060
 15. Caster- CP8040
 16. Axle Bearing (ea)- STDS1011
 17. Fork - CP1204
 18. Caster Axle- CP1204-A
 19. Charger- CPC
 When stock is gone use:- LRA402210
 20. Cross Frame 16"- CP8020-16
 Cross Frame 18"- CP8020-18
 Cross Frame 20"- CP8020-20
 Cross Frame 22"- CP8020-22
 Cross Frame 24"- CP8020-24
 21. Battery (ea) (2 needed)- LRA402208
 22. Motor Assembly Left- CP8050L
 Motor Assembly Right- CP8050R
 Motor Brush (ea)- CP5201</p> | <p>Motor Brush Cap (ea)- CP5202
 23. Side Frame Left- CPN8017L
 Side Frame Right- CPN8017R
 24. Rear Wheel- CP8012
 25. Battery Case 1 Plug- CP5211-1
 Battery Case 2 Plug- CP5211-2
 Battery Case Set - CP5115
 Battery Case Set with Batteries- CPLUSBATT
 Battery Strap- VENT-09
 26. Anti-Tipper (ea)- CP8025
 27. Brake Left- CP8035L
 Brake Right- CP8035R
 28. Fender Left- CPN801L
 Fender Right- CPN801R
 29. Folding Basket w/o Clips 16"- CP8015-16
 Folding Basket w/o Clips 18"- CP8015-18
 Folding Basket w/o Clips 20"- CP8015-20
 Folding Basket w/o Clips 22"- CP8015-22
 Folding Basket w/o Clips 24"- CP8015-24
 30. Basket Clip Rear- CP5005C
 Basket Clip Front- CP5005CF
 31. Seat Saddle- CPN8005
 32. Cross Frame Link 16"- CP8019-16
 Cross Frame Link 18"- CP8019-18
 Cross Frame Link 20"- CP8019-20
 Cross Frame Link 22"- CP8019-22
 Cross Frame Link 24"- CP8019-24
 33. Back Upholstery 16"- CP8006-16
 Back Upholstery 18"- CP8006-18
 Back Upholstery 20"- CP8006-20
 Back Upholstery 22"- CP8006-22
 Back Upholstery 24"- CP8006-24
 34. Tension Back 16"- CP8005-16
 Tension Back 18"- CP8005-18
 Tension Back 20"- CP8005-20
 Tension Back 22"- CP8005-22
 Tension Back 24"- CP8005-24
 35. Front Seat Saddle/Arm Receiver Lt.- CPN8008L
 Front Seat Saddle/Arm Receiver Rt.- CPN8008R
 36. Armrest Assembly Left- CPN800L
 Armrest Assembly Right- CPN800R
 37. Lower Arm Frame- CPN8014
 38. Adjustment Lock- CPN8009
 39. Full Arm Pad- STDSFLNB
 40. Upper Arm Frame- CPN8004
 41. Pivot Bracket- CPN8006
 42. Release Lever- CPN8007
 43. Back Cane- CPN8017
 Back Cane Adjustment Knob- STDS895
 44. Hub Cap- CPN8021
 45. Dust Cap- STDS1051</p> |
|---|---|