


ZOOME FLEX

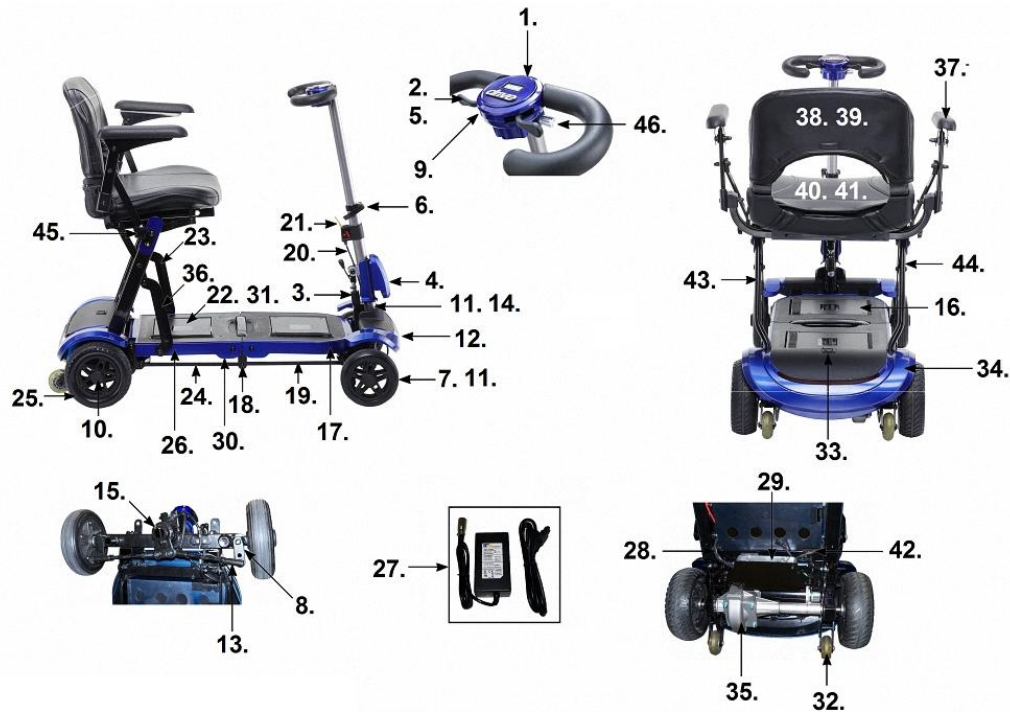

- | | |
|---|---|
| <ul style="list-style-type: none"> 1. Upper Panel Assembly Blue- FLEX-01
Upper Panel Assembly Blue Red- FLEX-01R 2. Throttle Assembly- FLEX-02 3. Gas Spring Assembly- FLEX-03 4. Front Reflector Assembly Blue- FLEX-04
Front Reflector Assembly Red- FLEX-04R 5. Throttle Potentiometer- FLEX-05 6. Quick Release Clamp- FLEX-06 7. Front Wheel- FLEX-07 8. Steering Knuckle Left- FLEX-08
Steering Knuckle Right- FLEX-09 9. On/Off Switch- FLEX-47 10. Hub Cap- FLEX-10 11. Wheel/Stem Bearing- FLEX-11 12. Front Shroud Blue- FLEX-12
Front Shroud Red- FLEX-12R 13. Tie Rods- FLEX-13 14. Handle Connector- FLEX-14 15. Steering Connector- FLEX-15 16. Battery Case Only- FLEX-16 17. Front Main Shroud Blue- FLEX-17
Front Main Shroud Red- FLEX-17R 18. Front Rotating Gear Assy.- FLEX-18 19. Front Support Rod- FLEX-19 20. Release Cable- FLEX-20 21. Release Lever- FLEX-21 22. Lithium Battery Pack- FLEX-22 23. Left Seat Support Assembly- FLEX-23 24. Rear Support Rod Assembly- FLEX-24 25. Rear Wheel- FLEX-25 26. Rear Main Shroud Blue- FLEX-26
Rear Main Shroud Red- FLEX-26R | <ul style="list-style-type: none"> 27. Battery Charger- FLEX-27 28. Power Harness- FLEX-28 29. Controller- FLEX-29 30. Rear Rotating Gear Assy.- FLEX-30 31. Battery Contacts (4 ea)- FLEX-31 32. Anti-Tip Wheel- FLEX-32 33. Charger Harness- FLEX-33 34. Rear Shroud Blue- FLEX-34
Rear Shroud Red- FLEX-34R 35. Motor, Transaxle & Brake- FLEX-35
Transaxle- FLEX-51
Motor- FLEX-52
Brake- FLEX-53 36. Right Seat Support Assembly- FLEX-36 37. Arm Pad & Screws- FLEX-37
Left Arm Assembly- FLEX-48
Right Arm Assembly- FLEX-49 38. Back Upholstery- FLEX-38 39. Back Cushion- FLEX-39 40. Seat Upholstery- FLEX-40 41. Seat Cushion- FLEX-41 42. Main Wiring Harness- FLEX-42 43. Left Hinge Cover Blue- FLEX-43
Left Hinge Cover Red- FLEX-43R 44. Right Hinge Cover Blue- FLEX-44
Right Hinge Cover Red- FLEX-44R 45. Armrest Locking Knob- FLEX-45
Substitute- LDR410115-03 46. Speed Controller- FLEX-46 n/s External Charging Docking Station- FLEX-50 |
|---|---|