


TITAN

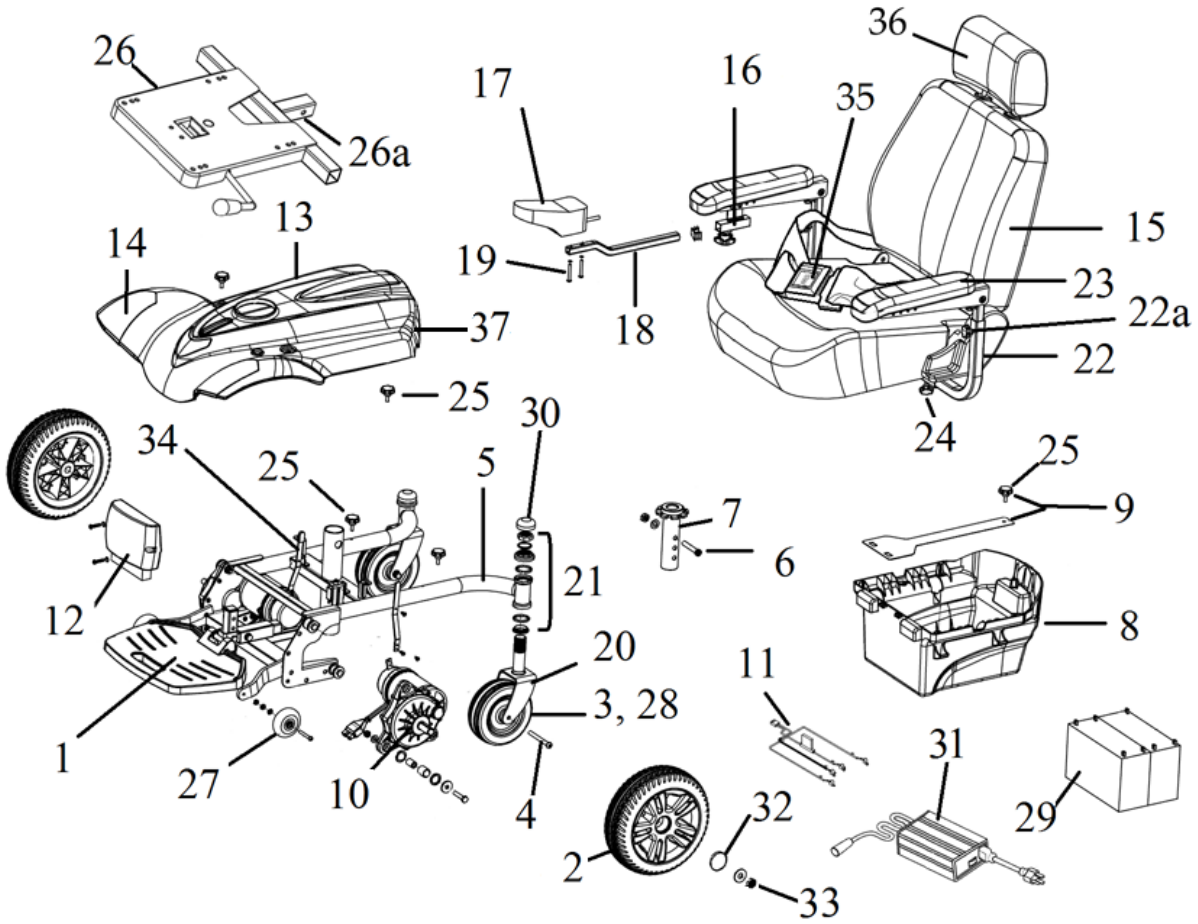

- |  |   |
|--|---|
| <p>1. Footplate &amp; Attaching Bracket- TITAN-01<br/>Footrest Depth Adj. Bolt- TITAN-14</p> <p>2. Drive Wheel Assembly- TITAN-11</p> <p>3. Caster Assembly- TITAN-03</p> <p>4. Caster Axle- TITAN-04</p> <p>5. Frame- TITAN-05</p> <p>6. Seat Post Bolt- TITAN-06</p> <p>7. Seat Post (1L Serial #-) TITAN-38</p> <p>8. Battery Tray Assembly- TITAN-07</p> <p>9. Battery Hold Down Strap- TITAN-08</p> <p>10. Motor &amp; Gear Box Left- TITAN-10<br/>Motor &amp; Gear Box Right- TITAN-09<br/>Axle Key- TITAN-40</p> <p>11. Battery Harness- TITAN-12<br/>Circuit Breaker- LRA402307</p> <p>12. Controller Shark- TITAN-13<br/>Control/Shroud Cable Shark- LPW182001-03<br/>Controller Linx- DLX-PM40-A-TITAN</p> <p>13. Shroud- TITAN-16</p> <p>14. Decoration Set Red- TITAN-17<br/>Decoration Set Blue- TITAN-18</p> <p>15. Captain's Seat Assembly-<br/>16"X16"X18"- TITAN-29<br/>16"X18"X18"- TITAN-30<br/>18"X16"X18"- TITAN-31<br/>18"X18"X16.5"- TITAN-19<br/>18"X18"X18"- TITAN-20<br/>20"X18"X18"- TITAN-21<br/>20"X20"X18"- TITAN-32<br/>20"X20"X19.5"- TITAN-33<br/>Pan Seat 18"- ST501-181818<br/>Pan Seat 20"- ST502-202020</p> <p>16. Joystick Bracket Knob- LDM400001-07<br/>Joystick Bracket - TITAN-24<br/>Joystick Bracket Sub.- LDM400001-05</p> <p>17. Joystick Shark- LPW182001<br/>Joystick Overlay Shark- DK-REMD01<br/>Joystick/Shroud Cable Shark- LPW182001-02<br/>Joystick Tip Shark- DC GJS0090<br/>Joystick Boot Shark- DC GJS0091<br/>Joystick Linx- LINX-JOYSTICK*<br/>Joystick/Control Cable Linx- GLM-BUS150-A<br/>Joystick Tip Linx- DC GPL55470</p> | <p>18. Joystick Arm- LDM400001-04</p> <p>19. Joystick Mount Screws (pr)- LDM400001-01</p> <p>20. Caster Fork- LDM400013</p> <p>21. Fork Stem Bearing- LRB402348</p> <p>22. Armrest Assembly Left- LDR410011L<br/>Armrest Assembly Right- LDR410011R</p> <p>22a. Armrest Height Knob- LDR410011-12</p> <p>23. Arm Pad- LDR410110<br/>Arm Pad Screws (pr)- LDR420018</p> <p>24. Armrest Locking Knob-<br/>1L Serial #- LDR410011-11<br/>VW Serial #- S31052</p> <p>25. Battery Tray/Shroud Knob-<br/>1L Serial #- LDR410115-03<br/>VW Serial #- VENT-86</p> <p>26. Seat Plate (1L Serial #-) LRC402414<br/>Seat Plate (VW Serial #-) C07-072-01001</p> <p>26a. Seat Plate Accessory Screw- LRC402414-02</p> <p>27. Anti-Tipper Wheel (ea)-<br/>1L Serial #- LDR420009-01<br/>VW Serial #- TITAN-41</p> <p>28. Caster Bearing- STDS1011</p> <p>29. Battery (ea) (2 needed)- LRA402208</p> <p>30. Dust Cap- LDR4110</p> <p>31. Charger 3 amp- LRA402209</p> <p>32. Hub Cap- LRM412202-HC</p> <p>33. Drive Wheel Nut &amp; Washer- TITAN-28</p> <p>34. Clutch Handle Assembly- TITAN-26<br/>Clutch Handle Tip- TITAN-27</p> <p>35. Seat Belt- STDS855-PW</p> <p>36. Headrest- ST-HEADREST</p> <p>37. Rear Reflectors (pr)- TITAN-37</p> <p>Not Shown-<br/>ELR Adapter- AE2500<br/>ELR- LK3JELR</p> |
|--|---|

\* Serial # VW017398 & higher or (21)2W1704202883 & higher.